VIASURE

Real Time PCR Detection Kit

***Mumps, Enterovirus &
Parvovirus***

CE IVD

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EN For download IFUS from other languages, please enter in certest.es/viasure/labeling. Once you be there, please following the instructions for access to the language that you need. If you need additional information, please contact: viasure@certest.es.

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Note: The user should notify the manufacturer and the competent authority of the Member State in which he is established as a user and/or patient of any serious incident related to the product.

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Content

1.	Intended use.....	8
2.	Summary and Explanation	8
3.	Principle of the procedure	9
4.	Reagents provided	9
5.	Reagents and equipment to be supplied by the user	9
6.	Transport and storage conditions.....	10
7.	Precautions for users	10
8.	Test procedure	12
8.1.	Specimen collection, transport and storage	12
8.2.	RNA extraction	12
9.	Result interpretation	13
9.1.	References with internal control (references in Annex 1 and 2)	13
9.2.	References with extraction control (references in Annex 3 and 4)	15
10.	Limitations of the test	17
11.	Quality control.....	18
12.	Performance characteristics.....	18
12.1.	Clinical sensitivity and specificity	18
12.2.	Analytical sensitivity	19
12.3.	Analytical specificity.....	20
12.4.	Analytical reactivity	21
	ANNEX 1	22
A1.1	Principle of the procedure.....	22
A1.2	Reagents provided.....	22
A1.3	Test procedure	24
A1.3.1	Lyophilized positive control.....	24
A1.3.2	PCR protocol	24
	ANNEX 2	25
A2.1	Principle of the procedure.....	25
A2.2	Reagents provided.....	25
A2.3	Test procedure	25
A2.3.1	Lyophilized positive control.....	25
A2.3.2	Lyophilized reaction mix tube.....	26
A2.3.3	PCR protocol	26
	ANNEX 3	27
A3.1	Principle of the procedure.....	27
A3.2	Reagents provided.....	27
A3.3	Test procedure	29
A3.3.1	Lyophilized extraction control	29
A3.3.2	Lyophilized positive control.....	29

A3.3.3 PCR protocol.....	29
ANNEX 4	31
A4.1 Principle of the procedure.....	31
A4.2 Reagents provided.....	31
A4.3 Test procedure.....	32
A4.3.1 Lyophilized extraction control	32
A4.3.2 Lyophilized positive control.....	32
A4.3.3 Lyophilized reaction mix tube.....	32
A4.3.4 PCR protocol.....	32

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ENGLISH

1. Intended use

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit is a real time RT-PCR assay designed for the qualitative detection and differentiation of RNA from Mumps virus, Enterovirus and/or Parechovirus in swabs specimens from vesicles, cutaneous skin, mouth, saliva, mucosal lesions (for Mumps and enterovirus), nasopharyngeal swabs and plasma samples (for Mumps) from patients with suspect of Mumps virus, Enterovirus and/or Parechovirus viral infection, by their healthcare professional (HCP). This test is intended to be used as an aid in the diagnosis of Mumps virus, Enterovirus and/or Parechovirus infection, in combination with clinical and epidemiological risk factors. RNA is extracted from clinical swabs specimens and plasma samples. Complementary DNA (cDNA) is synthesised and amplified using real time PCR and detected using fluorescent reporter dye probes specific of fusion protein (*F*) gene of Mumps, 5'UTR gene of Enterovirus, and 5'UTR gene of Parechovirus.

2. Summary and Explanation

The mumps virus is a RNA virus of the genus *Rubulavirus* in the family *Paramyxoviridae*. Mumps is a viral infection, which in its classical form causes acute parotitis (inflammation of the parotid salivary glands) and less frequently, orchitis, meningitis and pneumonia. Complications include sensorineural deafness, oligospermia, subfertility (rarely) and occasionally death from encephalitis. In the pre-vaccine era, mumps was primarily a childhood illness but epidemics among military recruits were not uncommon. The viral etiology of the disease was identified in 1934 and live attenuated mumps vaccines have been available since the 1960s. Most European countries have had routine childhood mumps immunization since the 1980s. Vaccination is now administered as a combination vaccination together with the measles and rubella attenuated virus components.

Enteroviruses are a group of viruses that cause a number of infectious illnesses which are usually mild. However, if they infect the central nervous system, they can cause serious illnesses like meningitis or acute flaccid paralysis (AFP). The many types of enteroviruses include the viruses that cause polio and hand, foot and mouth disease. Enteroviruses are most commonly spread by coming into contact with secretions like saliva, sputum or mucus of an infected person, or their faeces. There are currently no vaccines against non-polio enterovirus infection. Good hygiene practices, such as frequent handwashing, are essential to reducing the risk of becoming infected.

Human parechoviruses (HPeV) are widespread pathogens belonging to the *Picornavirus* family. Six genotypes, which have predominantly been isolated from children, are known. *Parechovirus* is a virus found in the respiratory and gastrointestinal tract of humans. Some people carry this around and have no symptoms while others, especially infants, can become unwell with a mild illness but in more severe cases the brain can become inflamed (known as meningitis), they become weak and lose consciousness. When someone is sick with parechovirus, they can spread the virus through infected respiratory secretions, saliva and faeces.

There is no vaccine to protect people from parechovirus infection. Good hygiene is the best protection.

There are many diagnostic methods available, but PCR-based assay is the most efficient test for the detection of these pathogens in a more sensitive and specific way.

3. Principle of the procedure

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit is designed for the diagnosis of Mumps virus, Enterovirus and/or Parechovirus infection swabs specimens (vesicles, cutaneous skin, mouth, saliva, mucosal lesions (for mumps and enterovirus) and nasopharyngeal swabs and plasma samples (for mumps). The detection is done in one step real time RT-PCR format where the reverse transcription and the subsequent amplification of specific target sequence occur in the same reaction well. The isolated RNA target is transcribed generating complementary DNA by reverse transcriptase, which is followed by the amplification of a conserved region of *F* (Fusion Protein) gene of Mumps virus, 5'UTR gene of Enterovirus, and 5'UTR gene of Parechovirus, using specific primers and a fluorescent-labelled probe.

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit is based on the 5'exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bounded to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal, which is proportional to the quantity of target template. This fluorescence can be measured on Real Time PCR platforms.

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer, polymerase and retrotranscriptase) in a stabilized format, as well as an **internal control** to verify the correct functioning of the amplification mix.

4. Reagents provided

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit includes the materials and reagents detailed in Annex 1 for open and rotor-gene format with internal control products, Annex 2 for tube format with internal control products, Annex 3 for open and rotor-gene format with extraction control products, and Annex 4 for tube format with extraction control products.

5. Reagents and equipment to be supplied by the user

The following list includes the materials that are required for use but not included in the VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler).
- Real Time PCR compatible plastic consumables (i.e. individual tubes, well-strips and/or microplates). Only for Tube format (Annex 2 and 4).
- RNA extraction kit.
- Collection and transport system.
- Laboratory freezers: -30°C to -10°C and/or ≤ -70°C.
- Centrifuge for 1.5mL tubes and PCR well strips or 96-well plate (if available).
- Vortex.
- Micropipettes (0.5-20 µL, 20-200 µL).
- Filter tips.
- Powder-free disposable gloves.

- Loading block (for use with Qiagen/Corbett Rotor-Gene® instruments). Only for Rotor Gene format (Annex 1 and 3).

VIASURE *Mumps*, *Enterovirus* & *Parechovirus* Real Time PCR Detection Kit has been validated on the following equipments: 7500 Fast Real-Time PCR System (Applied Biosystems), CFX96™ Real-Time PCR Detection System (Bio-Rad), Rotor-Gene® Q (Qiagen), AriaMx Real-Time PCR System (Agilent Technologies), DTprime Real-Time Detection Thermal Cycler (DNA-Technology), DTlite Real-Time PCR System (DNA-Technology), VIASURE V-Lab96 Cycler (CerTest Biotec S.L.), Cobas z 480 Analyzer (Roche Molecular Diagnostics) and QuantStudio™ 5 Real-Time PCR System (Applied Biosystems). To check thermocycler compatibility and most common detection channels consult website www.certest.es.

Optical measurement parameters of some thermocyclers must be adjusted to be suitable for operation with VIASURE Real Time PCR Detection Kits. This assay has been validated with the following set exposition values:

- DTprime Real-time Detection Thermal Cycler (RNA-Technology): FAM channel -500*, HEX channel – 1000, ROX channel – 1000 and Cy5 channel - 1000.
- DTlite Real-Time PCR System (RNA-Technology): FAM channel - 500, HEX channel - 500, ROX channel – 500 and Cy5 channel - 500.

*If the result in channel FAM is not as expected, there are no amplifications or high background noise is observed, please lower the exposure values indicated above to 150.

6. Transport and storage conditions

- The kits can be shipped and stored at 2-40°C until the expiration date which is stated on the label.
- Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles. Positive control has been validated as still being stable after 6 freeze-thaw cycles.
- Keep components away from light.
- For Tube format kits: Once the *Mumps*, *Enterovirus* & *Parechovirus* Reaction-Mix tube has been reconstituted, it may be kept it at 25°C±5°C or 2°C to 8°C for up to 4 hours. For a longer period of time, it is recommended store at -20°C and to separate in aliquots to minimize freeze and thaw cycles (up to 6 times).

7. Precautions for users

- The product is intended for use by qualified and trained clinical laboratory personnel specifically instructed and trained in the techniques of real-time PCR and *in vitro* diagnostic procedures (including training on the Real Time PCR instrument (thermocycler) and Nucleic acid extraction system).
- For *in vitro* diagnostic use.
- Do not use expired reagents and/or materials.
- Do not use the kit if the label that seals the outer box is broken.
- Do not use reagents if the protective box is open or broken upon arrival.
- Do not use reagents if the protective pouches are open or broken upon arrival.
- Do not use reagents if desiccant is not present or broken inside reagent pouches.
- Do not remove desiccant from reagent pouches once is open.

- Close protective pouches of reagents promptly with the zip seal after each use (for references: VS-MEP113L, VS-MEP113H, VS-MEP136, VS-MEP172, VS-MEP113LE, VS-MEP113HE, VS-MEP136E and VS-MEP172E). Remove any excess air in the pouches prior to sealing.
- Do not use reagents if the foil has been broken or damaged.
- Do not mix reagents from different pouches and / or kits and / or lots and / or another supplier.
- Protect reagents against from humidity. Prolonged exposure to humidity may affect product performance.
- For references VS-MEP101, VS-MEP136, VS-MEP172, VS-MEP101E, VS-MEP136E and VS-MEP172E (compatible for use with Qiagen/Corbett Rotor-Gene® instruments) use the loading block to pipette reagents and samples into each tube and to help with fitting caps properly and avoid cross contamination.
- An appearance of the reaction mixture in stabilized format, normally found at the bottom of the tube, different from the usual one (without conical shape, inhomogeneous, smaller/larger in size and/or colour different from whitish) does not alter the functionality of the test.
- Design a unidirectional workflow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment, and reagents to the area in which the previous step was performed. Use separate areas for the preparation of patient samples and controls to prevent false positive results.
- In cases where other PCR tests are conducted in the same general area of the laboratory, care must be taken to ensure that the VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit and any additional reagents or equipment required for testing are not contaminated. Always avoid microbial and ribonuclease (RNase)/deoxyribonuclease (DNase) contamination of reagents. The use of sterile RNase/DNase-free disposable aerosol resistant or positive displacement pipette tips is recommended. Use a new tip for each specimen. Gloves must be changed before manipulating reagents.
- Follow Good Laboratory Practices. Wear protective clothing, use disposable gloves, goggles, and mask. Do not eat, drink, or smoke in the working area. Wash your hands after finishing the test.
- Specimens must be treated as potentially infectious, as well as all the reagents and materials that have been exposed to the samples and they must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment, and disposal of samples.
- Samples and reagents must be handled in a biological safety cabinet. Use personal protective equipment (PPE) consistent with current guidelines for the handling of potentially infectious samples. Dispose waste in compliance with local and state regulations.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.
- In accordance with Regulation (EC) No 1907/2006 (REACH), VIASURE Real Time PCR Detection Kits do not require Material Safety Data Sheets on account of their classification as non-hazardous to health and the environment because they do not contain substances and/or mixtures which meet the hazard classification criteria available in Regulation (EC) No 1272/2008 (CLP) or which are in concentrations higher than the value established in the mentioned regulation for their declaration.
- Consult each Real Time PCR instrument's reference manual for additional warnings, precautions, and procedures.

8. Test procedure

Please see Annex 1 for open and rotor-gene format with internal control products Test Procedure, Annex 2 for Tube format with internal control products Test Procedure, Annex 3 for open and rotor-gene format with extraction control products Test Procedure and Annex 4 for Tube format with extraction control products Test Procedure.

8.1. Specimen collection, transport and storage

The VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit has been tested in swabs specimens (vesicles, cutaneous skin, mouth, saliva, mucosal ulcer lesions (for mumps and enterovirus detection)), in nasopharyngeal swabs and plasma samples (for mumps detection), and in cerebrospinal fluid (CSF) (for enterovirus detection). Swabs specimens were collected in M4RT (Viral Transport Media) or in cobas® PCR Media Dual Swab Sample Kit. Other types of samples must be validated by the user.

Collection, storage, and transport of specimens should be maintained per the conditions validated by the user. Overall, clinical samples should be collected and labelled appropriately in clean containers with or without transport media (depending on sample type) and processed as soon as possible to guarantee the quality of the test. The specimens should be processed immediately or transported at -20°C or lower, following the local and national regulations for the transport of pathogen material, if preservation medium permits. It is recommended to use fresh specimens for the test. The samples can be stored frozen at -20°C or ideally at -80°C for conservation **if preservation medium permits**. We recommend following appropriate laboratory guidelines and/or manufacturer's recommendations for proper preservation of samples in the selected collection and transport medium. Repeated freeze-thaw cycles should be avoided in order to prevent degradation of the sample and nucleic acids.

The clinical specimens must be collected, transported, and stored according to appropriate laboratory guidelines. For details, refer to the IDSA guideline (Miller, J. M., Binnicker, M. J., Campbell, S., ... & Pritt, B. S. (2018). A guide to utilization of the microbiology laboratory for diagnosis of infectious diseases: 2018 update by the Infectious Diseases Society of America and the American Society for Microbiology. *Clinical Infectious Diseases*, 67(6), e1-e94), and García-Lechuz Moya, J.M., González López, J.J., Orta Mira, N., Sánchez Romero, M.I. (2017). Recogida, transporte y procesamiento general de las muestras en el Laboratorio de Microbiología. 2017. 1b. Sánchez Romero, M.I., (coordinadora). Procedimientos en Microbiología Clínica. Cercenado Mansilla, E., Cantón Moreno, R., (editores). *Sociedad Española de Enfermedades Infecciosas y Microbiología Clínica (SEIMC)*.

8.2. RNA extraction

Perform the sample preparation according to the recommendations appearing in the instructions for use of the extraction kit used.

Due to VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit is also available with extraction control (references include in Annexes 3 and 4), if the Extraction Control (EC) is used to monitor nucleic acid isolation and as PCR inhibition control, add 5µL of the reconstituted EC to the specimen and/or lysis buffer mixture (clinical specimen, as well as positive control and/or negative control). Close each tube and vortex for 10 seconds. If the Extraction Control is used only as a PCR inhibition control, 1µL of the reconstituted EC should be added to the reconstituted Reaction-Mix.

For RNA extraction from clinical swabs, plasma samples and CSF, you can use your manual or automatic routine optimized system, or any commercially available RNA extraction kit and follow the manufacturer's instructions. The following extraction kits have been validated:

- MagDEA Dx SV kit, using the magLEAD® 12gC instrument (Precision System Science Co.).
- NucliSENS® eMAG® (Biomérieux).
- Invisorb® Spin Universal Kit (Invitex).

9. Result interpretation

9.1. References with internal control (references in Annex 1 and 2)

All the result of the test should be evaluated by a healthcare professional in the context of medical history, clinical symptoms, and other diagnostic tests. Check Internal Control (IC) signal to verify the correct functioning of the amplification mix. The analysis of the controls and samples is done by the software of the used real time PCR equipment itself according to manufacturer's instructions.

It is recommended to set the threshold values for each channel (target) independently by the end-user. Use the Positive Control amplification curve as a starting point during the run validation (before than interpretation of patient sample results), in order to ensure that thresholds fall within the exponential phase of the fluorescence curves and above any background signal. The threshold value for different instruments may vary due to different signal intensities.

The use of positive and negative controls in each run validates the reaction by checking the absence of signal in the negative control well and the presence of signal for Mumps virus, Enterovirus and Parechovirus in the positive control well.

For a valid diagnostic test run, the following control conditions must be met:

Controls	Mumps virus (FAM) ¹	Enterovirus (HEX) ¹	Parechovirus (ROX) ¹	Internal Control (Cy5) ²	Interpretation of Controls
Positive Control (PC)	≤40	≤40	≤40	≤40	Valid
Negative Control (NC)	>40 or no signal	>40 or no signal	>40 or no signal	≤40	Valid

Table 1. Expected Performance of Controls. Ct values. no signal = no amplification curves.

1 In cases where either or both of the control assays have failed (an amplification signal is observed in the negative control and/or signals absence in the positive control well for any target channel), all results are reported as 'Invalid' and retesting is required.

2. The Internal Control (IC) should show an amplification signal (Ct ≤40) in control wells (PC and NC).

Assessment of clinical samples test results should be performed after the positive and negative controls have been examined and determined to be valid and acceptable. If one or more controls are not valid, the patient results cannot be interpreted.

For interpretation of individual patient sample results, use the following table, read, and analyse the results:

Mumps virus (FAM)	Enterovirus (HEX)	Parechovirus (ROX)	Internal control (Cy5)	Interpretation for patient's individual samples	
≤40	> 40 or no signal	> 40 or no signal	> 40 or no signal ¹	Valid	Mumps virus RNA Detected
> 40 or no signal	≤40	> 40 or no signal	> 40 or no signal ¹	Valid	Enterovirus RNA Detected
> 40 or no signal	> 40 or no signal	≤40	> 40 or no signal ¹	Valid	Parechovirus RNA Detected
≤40	≤40	> 40 or no signal	> 40 or no signal ¹	Valid	Mumps virus and Enterovirus RNA Detected
≤40	> 40 or no signal	≤40	> 40 or no signal ¹	Valid	Mumps virus and Parechovirus RNA Detected
> 40 or no signal	≤40	≤40	> 40 or no signal ¹	Valid	Enterovirus and Parechovirus RNA Detected
≤40	≤40	≤40	> 40 or no signal ¹	Valid	All targets' RNA Detected
>40 or no signal	>40 or no signal	>40 or no signal	≤35 ²	Valid	Targets not Detected
>40 or no signal	>40 or no signal	>40 or no signal	>35 or no signal ²	Invalid	Test Failure – Repeat Testing ²

Table 2. Interpretation of individual patient sample results. Ct values. no signal = no amplification curves.


1 The Internal Control (IC) shows or not an amplification signal (Ct ≤40 or no signal). Sometimes, its detection is not necessary because a high copy number of the target can cause preferential amplification of target-specific nucleic acids.

2 In the case of Mumps virus, Enterovirus and Parechovirus target genes negative, IC must show an amplification signal with Ct ≤ 35. If there is an absence of signal or Ct value > 35 of the Internal Control, the result is considered as 'Invalid', and retesting is required. It is recommended to repeat the qPCR by diluting the RNA sample 1:10 and/or 1:100, or re-extract and retest to check for possible failure in the extraction procedure and/or inhibition issues.

In case of a continued ambiguous result, it is recommended to review the instructions for use; the extraction process used by the user; to verify the correct performance of each RT-qPCR steps and review the parameters; and to check the sigmoid shape of the curve and the intensity of fluorescence. It is also recommended to repeat the assay, preferably in duplicate. Depending on the available material:

- repeat RT-qPCR with the same isolated RNA sample, or
- re-extract and retest another aliquot of the same specimen or,
- obtain a new specimen and retest.

Figure 1. Correct run of negative and positive control run on the CFX96™ Real-Time PCR Detection System (Bio-Rad).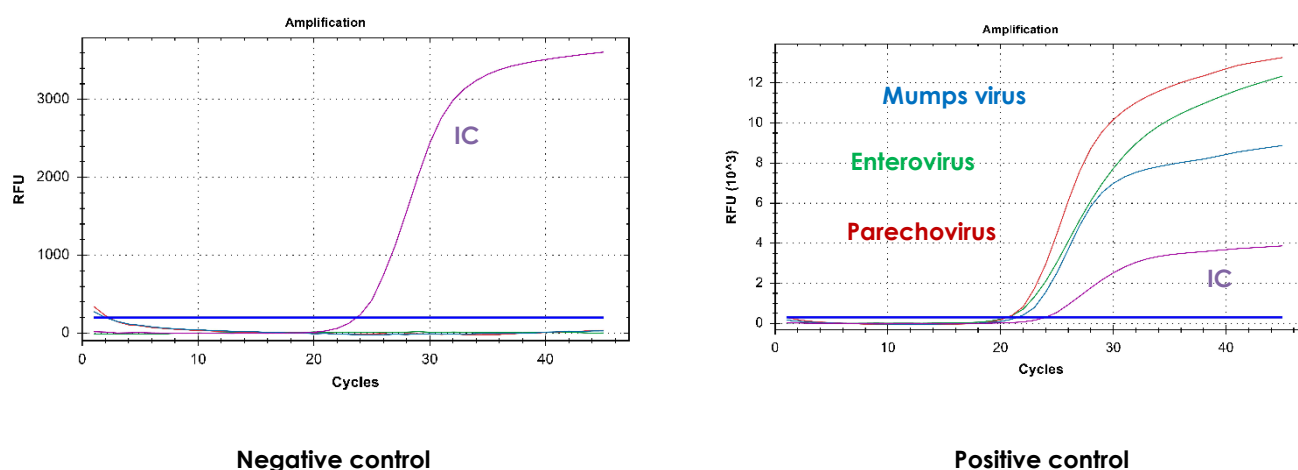

9.2. References with extraction control (references in Annex 3 and 4)

All the results of the test should be evaluated by a healthcare professional in the context of medical history, clinical symptoms, and other diagnostic tests. Check Extraction Control (EC) signal to verify the correct functioning of the amplification mix. The analysis of the controls and samples is done by the software of the used real time PCR equipment itself according to manufacturer's instructions.

It is recommended to set the threshold values for each channel (target) independently by the end-user. Use the Positive Control amplification curve as a starting point during the run validation (before than interpretation of patient sample results), in order to ensure that thresholds fall within the exponential phase of the fluorescence curves and above any background signal. The threshold value for different instruments may vary due to different signal intensities.

The use of positive and negative controls in each run validates the reaction by checking the absence of signal in the negative control well and the presence of signal Mumps virus, Enterovirus and Parechovirus in the positive control well. For a valid diagnostic test run, the following control conditions must be met:

Controls	Mumps virus (FAM) ¹	Enterovirus (HEX) ¹	Parechovirus (ROX) ¹	Extraction Control (Cy5) ²	Interpretation of Controls
Positive Control (PC)	≤40	≤40	≤40	≤40	Valid
Negative Control (NC)	>40 or no signal	>40 or no signal	>40 or no signal	≤40	Valid

Table 3. Expected performance of controls. Ct values. no signal = no amplification curves.

1 In cases where either or both of the control assays have failed (an amplification signal is observed in the negative control and/or signals absence in the positive control well for any target channel), all results are reported as 'Invalid' and retesting is required.

2 The Extraction Control (EC) should show an amplification signal (Ct ≤40) in control wells (PC and NC).

Assessment of clinical samples test results should be performed after the positive and negative controls have been examined and determined to be valid and acceptable. If one or more controls are not valid, the patient results cannot be interpreted.

For interpretation of individual patient sample results, use the following criteria for read and analyze the results:

Mumps virus (FAM)	Enterovirus (HEX)	Parechovirus (ROX)	Extraction control (Cy5)	Interpretation for patient's individual samples	
≤40	> 40 or no signal	> 40 or no signal	> 40 or no signal ¹	Valid	Mumps virus RNA Detected
> 40 or no signal	≤40	> 40 or no signal	> 40 or no signal ¹	Valid	Enterovirus RNA Detected
> 40 or no signal	> 40 or no signal	≤40	> 40 or no signal ¹	Valid	Parechovirus RNA Detected
≤40	≤40	> 40 or no signal	> 40 or no signal ¹	Valid	Mumps virus and Enterovirus RNA Detected
≤40	> 40 or no signal	≤40	> 40 or no signal ¹	Valid	Mumps virus and Parechovirus RNA Detected
> 40 or no signal	≤40	≤40	> 40 or no signal ¹	Valid	Enterovirus and Parechovirus RNA Detected
≤40	≤40	≤40	> 40 or no signal ¹	Valid	All targets' RNA Detected
>40 or no signal	>40 or no signal	>40 or no signal	≤35 ²	Valid	Targets not Detected
>40 or no signal	>40 or no signal	>40 or no signal	>35 or no signal ²	Invalid	Test Failure – Repeat Testing ²

Table 4. Interpretation of individual patient sample results. Ct values. no signal = no amplification curve.


1 The Extraction Control (EC) shows or not an amplification signal (Ct ≤40 or no signal). Sometimes, its detection is not necessary because a high copy number of the target can cause preferential amplification of target-specific nucleic acids. Differences can be observed in the values of Ct in the extraction controls between the controls and the clinical samples, due to the extraction process.

2 In the case of Mumps virus, Enterovirus and Parechovirus target genes negative, EC must show an amplification signal with Ct ≤35. If there is an absence of signal or Ct value > 35 of the Extraction Control, the result is considered as 'Invalid', and retesting is required. It is recommended to repeat the RT-qPCR by diluting the RNA sample 1:10 and/or 1:100, or re-extract and retest to check for possible failure in the extraction procedure and/or inhibition issues.

In case of a continued ambiguous result, it is recommended to review the instructions for use, the extraction process used by the user, to verify the correct performance of each RT-qPCR steps and review the parameters, and to check the sigmoid shape of the curve and the intensity of fluorescence. It is also recommended to repeat the assay, preferably in duplicate. Depending on the available material:

- repeat RT-qPCR with the same isolated RNA sample, or
- re-extract and retest another aliquot of the same specimen or,
- obtain a new specimen and retest.

Figure 2. Correct run of negative and positive control run on the CFX96™ Real-Time PCR Detection System (Bio-Rad).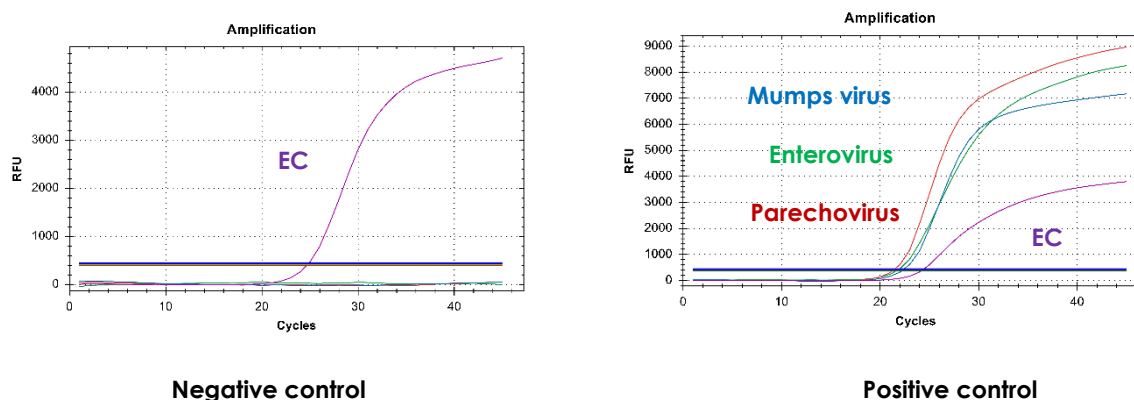

10. Limitations of the test

- The results of the test should be evaluated by a health care professional in the context of medical history, clinical symptoms, and other diagnostic tests.
- Although this assay can be used with other types of samples, it has been validated only with RNA extracted from swabs specimens (vesicles, cutaneous skin, mouth, saliva, mucosal ulcer/lesions (for mumps and enterovirus)), from nasopharyngeal swabs and plasma samples (for mumps), and from CSF (for enterovirus).
- The quality of the test depends on the quality of the sample; nucleic acid must be properly extracted from clinical samples.
- This test is a qualitative test and does not provide quantitative values or indicate the number of organisms present.
- Extremely low levels of target below the limit of detection might be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Mumps virus, Enterovirus and/or Parechovirus, either the high number of cDNA template copies which contains each *Mumps*, *Enterovirus* & *Parechovirus* Positive Control vial, samples containing high concentrations of target RNA or contamination due to PCR products from previous reactions.
- There is a possibility of false positive results due to cross-contamination between Extraction Control and *Mumps*, *Enterovirus* & *Parechovirus* Positive Control, which contains high copies template, during their reconstitution by adding of Water RNase/DNase free (white vial). Each procedure must take place in established order and in separate laboratory areas.
- False Negative results may arise from several factors and their combinations, including:
 - Improper specimens' collection, transport, storage, and/or handling methods.
 - Improper processing procedures (including RNA extraction).
 - Degradation of the viral RNA during sample shipping/storage and/or processing.
 - Mutations or polymorphisms in primer or probe binding regions may affect detection of new or unknown Mumps virus, Enterovirus and/or Parechovirus variants.
 - A viral load in the specimen below the limit of detection for the assay.
 - The presence of qPCR inhibitors or other types of interfering substances. The impacts of vaccines, antiviral therapeutics, antibiotics, chemotherapeutics, or immunosuppressant drugs, used to prevent the Mumps virus, Enterovirus and/or Parechovirus infection, or used during the treatment of the infection have not been evaluated.
 - Failure to follow instructions for use and the assay procedure.
- A positive test result does not necessarily indicate the presence of viable viruses and does not imply that these viruses are infectious or are the causative agents for clinical symptoms. However, a positive result is indicative of the presence of target viral sequences.
- Negative results do not preclude Mumps virus, Enterovirus and/or Parechovirus infection, and should not be used as the sole basis for treatment or other patient management decisions. Optimum specimen types and timing for peak viral levels during infections caused by Mumps virus, Enterovirus and/or Parechovirus have not been determined. The collection of multiple specimens (types and time points) from the same patient may be necessary to detect the pathogen.

- If diagnostic tests for other viral infection are negative and the patient's clinical presentation and epidemiological information suggest that Mumps virus, Enterovirus and/or Parechovirus infection is possible, then a false negative result should be considered, and a re-testing of the patient should be discussed.
- Fluorescence values may vary due to multiple factors such as: PCR equipment, extraction system, type of sample, previous treatment of the sample, etc... among others.

11. Quality control

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) or the Extraction control (EC) in each well confirms the correct performance of the technique.

12. Performance characteristics

12.1. Clinical sensitivity and specificity

The clinical performance of VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit was tested using clinical samples (swabs specimens from vesicles, cutaneous skin, mouth, saliva, mucosal ulcer/lesions (for mumps and enterovirus), nasopharyngeal swabs and plasma samples (for mumps), and cerebrospinal fluid (CSF) (for enterovirus)) from patients with suspected of viral infection. In order to determine the clinical diagnostic accuracy, different multicenter evaluations have been conducted through collaboration with national and international entities. A summary of the sites, sample type and workflow are included in the following table. The results were as follows:

	Site	Sample type	Workflow	Target
1	Hospital Clinico Universitario Lozano Blesa (HCULB) (Zaragoza, Spain)	CSF	MagDEA Dx SV kit using the magLEAD® 12gC instrument (Precision System Science Co.)* + DTLite Real-Time PCR System (DNA-Technology)	Enterovirus
2	National Reference Center for Emerging Viral Infections (CRIVE), Laboratory of Virology of Geneva University Hospital (HUG) (Geneva, Switzerland)	Nasopharyngeal swabs and plasma samples	NucliSENS® eMAG® (Biomérieux) + QuantStudio™ 5 Real-Time PCR System (Applied Biosystems)	Mumps virus
3	CerTest Biotec facilities (Zaragoza, Spain) in collaboration with Eurofins Biomnis and CerBa Xpert.	Swab specimens (vesicles, cutaneous-skin, mouth, mucosal ulcer/lesions, mouth-saliva)	MagDEA Dx SV kit using the magLEAD® 12gC instrument (Precision System Science Co.) + CFX96™ Real-Time PCR Detection System (Bio-Rad)	Mumps virus
				Enterovirus

Table 5. Site, sample type, workflow, and target.

* Clinical evaluation with CSF was also performed without the step of nucleic acid extraction. The same CT value was obtained in the 48 specimens.

True positive and negative values, false positive and negative values, sensitivity, specificity, PPV, NPV values for VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit were calculated in relation to each comparator assay as shown in the following table:

Site	Comparator assay	Target	TP	TN	FP	FN	Sensitivity	Specificity	PPV	NPV
1	CLART® ENTHERPEX (Genomica)	Enterovirus	3	45	0	0	1 (0.29 – 1)	1 (0.92 – 1)	1 (0.29 – 1)	1 (0.92 – 1)
2	Validated in-house Real Time PCR	Mumps virus	7	13	0	0	1 (0.59 – 1)	1 (0.75 – 1)	1 (0.59 – 1)	1 (0.75 – 1)
3	COFRAC-accredited in-house real-time RT-PCR	Mumps virus	4	107	0	0	1 (0.39 – 1)	1 (0.96 – 1)	1 (0.39 – 1)	1 (0.96 – 1)
		Enterovirus	13	98	0	0	1 (0.75-1)	1 (0.96 – 1)	1 (0.75-1)	1 (0.96 – 1)

Table 6. True positive (TP) and negative values (TN), false positive (FP) and false negative (FN) values, sensitivity, specificity, Predictive Positive Values (PPV), Predictive Negative Values (NPV) for VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit.

Results show high agreement to detect Mumps virus and Enterovirus using VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit.

12.2. Analytical sensitivity

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit has a detection limit of 10 cDNA copies per reaction for Mumps virus, Enterovirus and Parechovirus, with a positive rate of 95%.

Figure 3. Dilution series of Mumps virus (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel FAM).

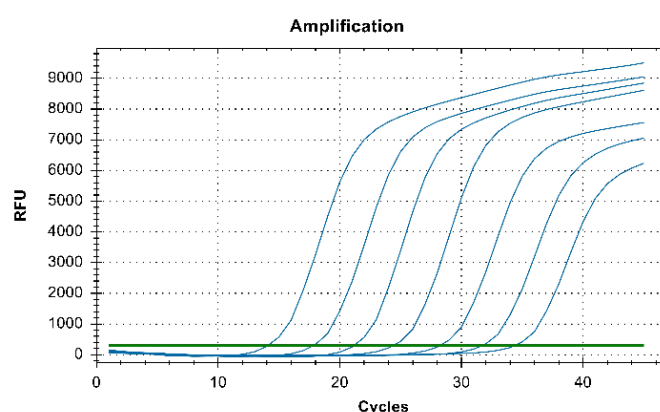
Figure 4. Dilution series of Parechovirus (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel ROX).

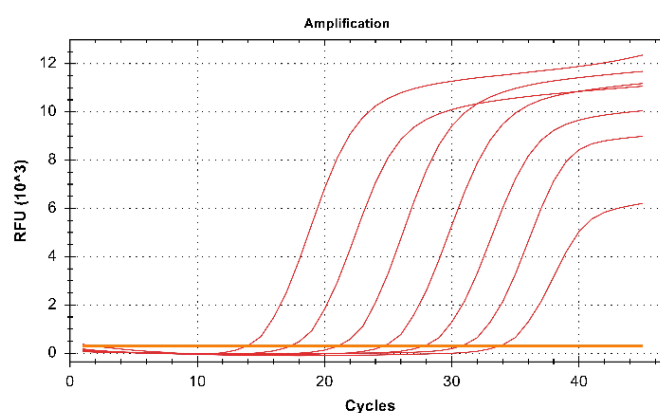
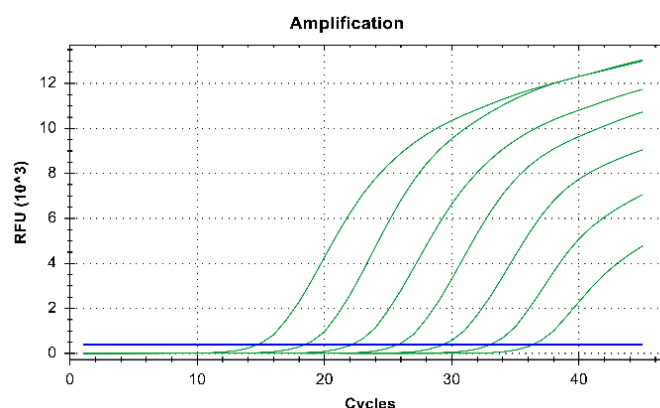
Figure 5. Dilution series of Enterovirus (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel HEX).

12.3. Analytical specificity

The specificity of VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit was confirmed by testing a panel of different microorganisms associated to diseases in immunosuppressed patients, such as those who received transplants, or those microorganisms that are capable of producing meningitis. No cross-reactivity was detected between any of the following microorganisms tested.

Cross-reactivity testing					
<i>Acinetobacter baumannii</i>	-	Epstein-Barr virus	-	<i>Plasmodium falciparum</i> 3D7	-
Adenovirus types 1, 2, 3, 4, 5, 8, 15, 31, 40 and 41	-	<i>Escherichia coli</i> 0.1285; O18:H7:K1	-	<i>Pneumocystis jirovecii</i> Type A1	-
<i>Aspergillus fumigatus</i>	-	<i>Haemophilus influenzae</i> MinnA	-	<i>Pneumocystis jirovecii</i> g885652	-
<i>Bacteroides fragilis</i>	-	Hepatitis A virus	-	<i>Pseudomonas aeruginosa</i>	-
BK Virus types Ib-2 and IV	-	Herpes simplex virus 1 (HSV-1) strain MacIntyre	-	St Louis Encephalitis Virus	-
<i>Candida albicans</i>	-	Herpes simplex virus 2 (HSV-2) MS	-	<i>Staphylococcus aureus</i> subsp. <i>aureus</i>	-
Chikungunya S27 Petersfield	-	Human herpesvirus 6 (HHV6) Strain Z29	-	<i>Streptococcus agalactiae</i> Z019	-
Cytomegalovirus AD-169	-	Human herpesvirus 6 (HHV6) Type A	-	<i>Streptococcus pneumoniae</i> Z022	-
<i>Cryptococcus gatti</i> Z156	-	Human herpesvirus 6 (HHV6) Type B	-	Tick-Borne encephalitis virus strain Neudorfl	-
Dengue serotype 1 Hawaii A	-	JC Virus Type 1A	-	<i>Toxoplasma gondii</i> type II	-
Dengue serotype 2 New guinea C	-	JC Virus Type 2B	-	<i>Treponema pallidum</i>	-
Dengue serotype 3 H87	-	<i>Klebsiella oxytoca</i>	-	Varicella-Zoster Virus OKA	-
Dengue serotype 4 H241	-	<i>Listeria innocua</i> serotype 6a	-	Varicella-Zoster Virus 9/84	-
Japanese Encephalitis virus	-	<i>Listeria ivanovii</i> subsp. <i>Ivanovii</i> serovar 5	-	Varicella-Zoster Virus Ellen	-
<i>Enterobacter aerogenes</i> serotype Cloaca B	-	<i>Listeria monocytogenes</i> serovar 4b	-	West Nile virus Heja	-

Cross-reactivity testing					
<i>Enterobacter cloacae</i> serotype Cloaca A	-	<i>Listeria monocytogenes</i> serotype 1/2b	-	West Nile virus NY99	-
<i>Enterococcus durans</i>	-	Measles virus	-	West Nile virus Ug37	-
<i>Enterococcus faecalis</i>	-	<i>Neisseria meningitidis</i> serogroup A	-	Yellow Fever Virus strain 17D	-
<i>Enterococcus faecium</i> serotype 11	-	Parvovirus B19	-	<i>Yersinia enterocolitica</i> O:3	-

Table 7. Reference pathogenic microorganisms used in this study.

12.4. Analytical reactivity

The reactivity of VIASURE *Mumps*, *Enterovirus* & *Parechovirus* Real Time PCR Detection Kit for Mumps virus was evaluated against RNA extracted from Mumps virus strain Enders (as template) showing positive results.

The reactivity of VIASURE *Mumps*, *Enterovirus* & *Parechovirus* Real Time PCR Detection Kit for Enterovirus was evaluated against RNA extracted from Enterovirus Echovirus Types 6, 7, 11, 18, 25, 30, Human Echovirus 4 strain Piascek; Enterovirus types 68, D68, D68 B3, 71 and A71, Enterovirus Coxsackievirus types A4, A9, A16, A21, A24, B3 and B4, Human Coxsackievirus B1 strain Conn-5 and Enterovirus D68 strain US/MO/14-18949 (as templates), showing positive results.

The reactivity of VIASURE *Mumps*, *Enterovirus* & *Parechovirus* Real Time PCR Detection Kit for Parechovirus was evaluated against RNA extracted from Parechovirus types 1, 2, 3, 4, 6 and Human Parechovirus 1 strain Harris (as templates), showing positive results.

ANNEX 1

OPEN FORMAT AND ROTOR-GENE FORMAT WITH INTERNAL CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 1 x 8-well strips, low profile	VS-MEP101L
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 1 x 8-well strips, high profile	VS-MEP101H
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 6 x 8-well strips, low profile	VS-MEP106L
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 6 x 8-well strips, high profile	VS-MEP106H
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 12 x 8-well strips, low profile	VS-MEP112L
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 12 x 8-well strips, high profile	VS-MEP112H
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 96-well plate, low profile	VS-MEP113L
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 96-well plate, high profile	VS-MEP113H
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 2 x 4-well strips, Rotor-Gene®	VS-MEP101
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 9 x 4-well strips, Rotor-Gene®	VS-MEP136
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 18 x 4-well strips, Rotor-Gene®	VS-MEP172

Table A1 1. References

A1.1 Principle of the procedure

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer, polymerase and retrotranscriptase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity.

Target	Channel	Gene
Mumps virus	FAM	F gene
Enterovirus	HEX, VIC or JOE *	5'UTR gene
Parechovirus	ROX	5'UTR gene
Internal control (IC)	Cy5	-

Table A1 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult the website www.cerfetest.es.

A1.2 Reagents provided

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit includes the following materials and reagents detailed in Tables A1.3, A1.4 and A1.5. Based on the commercial presentation and the Real Time PCR platform used, the stabilized PCR reaction mix could be placed inside different wells and could be marketed on multiple formats. Table A1.3 includes materials and reagents to be used with 8-well strips compatible devices. Table A1.4 includes materials and reagents to be used with 96-well plate compatible devices. Table A1.5 includes

materials and reagents for use with Qiagen/Corbett Rotor-Gene® instruments for 4-well strips. (Consult the thermocycler compatibility on CerTest's website www.cerTEST.es).

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1/6/12 x 8-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	1/6/12 x 8-cap strip

Table A1 3. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref. VS-MEP101L, VS-MEP101H, VS-MEP106L, VS-MEP106H, VS-MEP112L and VS-MEP112H.

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 x 8-cap strip

Table A1 4. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref VS-MEP113L and VS-MEP113H.

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 4-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	Transparent	2/9/18 x 4-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
4-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	2/9/18 X 4-cap strip

Table A1 5. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref. VS-MEP101, VS-MEP136 and VS-MEP172. For use with Qiagen/Corbett Rotor-Gene® instruments and compatible accessories with strips of 4 tubes 0.1 ml (72-Well Rotor and Locking Ring 72-Well Rotor).

A1.3 Test procedure

A1.3.1 Lyophilized positive control

Mumps, Enterovirus & Parechovirus Positive Control contains high copies of the template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Mumps, Enterovirus & Parechovirus* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A1.3.2 PCR protocol

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run for each assay. Peel off protective aluminium seal from plates or strips.

- 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

- 2) Adding samples and controls.

Add 5 µL of RNA sample, reconstituted *Mumps, Enterovirus & Parechovirus* Positive Control (red vial) or Negative Control (violet vial) in different wells and close them with the provided caps. It is recommended to briefly centrifuge the 8-well strips or 96-well plate, or gently tap each strip onto a hard surface to ensure that all the liquids are at the bottom of the tubes (for Qiagen/Corbett Rotor-Gene®).

Load the plate or the strips in the thermocycler.

- 3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A1 6. PCR protocol

Fluorogenic data should be collected during the extension step (*) through the FAM (*Mumps virus*), ROX (*Parechovirus*), HEX, JOE or VIC (*Enterovirus*) and Cy5 (Internal Control (IC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 2

TUBE FORMAT WITH INTERNAL CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE <i>Mumps, Enterovirus & Parechovirus</i> Real Time PCR Detection Kit, 4 tubes x 24 reactions	VS- MEP196T

Table A2. 1. References.

A2.1 Principle of the procedure

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit contains in each Reaction-Mix tube all the components necessary for 24 real time PCR reactions (specific primers/probes, dNTPs, buffer, polymerase and retrotranscriptase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity.

Target	Channel	Gene
Mumps virus	FAM	<i>F</i> gene
Enterovirus	HEX, VIC or JOE *	<i>5'UTR</i> gene
Parechovirus	ROX	<i>5'UTR</i> gene
Internal control (IC)	Cy5	-

Table A2. 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, channel, to check most common detection channels consult website www.ceratest.es.

A2.2 Reagents provided

VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table A2.3.

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> Reaction-Mix tube	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and internal control in stabilized format	White	4 vials
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL

Table A2. 3. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref. VS-MEP196T.

A2.3 Test procedure

A2.3.1 Lyophilized positive control

Mumps, Enterovirus & Parechovirus Positive Control contains high copies of the template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the

lyophilized *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A2.3.2 Lyophilized reaction mix tube

Determine the number of required reactions including samples and controls (one positive and negative control must be included in each run). Obtain the correct number of lyophilized Reaction-Mix vials (24-reactions each one) for testing.

Recommendation is to open and manipulate the *Mumps*, *Enterovirus* & *Parechovirus* Reaction-Mix tube in pre-PCR laboratory area. Open lyophilized Reaction-mix tube (white vial) carefully to avoid disruption of the pellet and add 390 µL of Rehydration Buffer (blue vial) supplied. Mix gently by pipetting up and down. Spin down briefly to remove bubbles generated during mixing.

Once the Reaction-Mix tube has been re-suspended, return unused reagents to the appropriate storage conditions at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The volume of the rehydrated Reaction-Mix is sufficient for 24 reactions. The rehydrated Reaction-Mix may be kept at 25°C±5°C or 2-8°C for up to 4-hours (see Transport and storage conditions section for additional storage options).

A2.3.3 PCR protocol

- 1) Adding rehydrated Reaction-Mix to the number of required wells.

Add 15 µL of rehydrated *Mumps*, *Enterovirus* & *Parechovirus* Reaction-Mix (white vial) into each tube.

- 2) Adding samples and controls.

Add 5 µL of RNA sample, reconstituted *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the tubes with caps or seal the plate. Centrifuge briefly.

Load the plate, the strips, or tube in the thermocycler.

- 3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A2. 4. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Mumps* virus), ROX (*Parechovirus*), HEX, JOE or VIC (*Enterovirus*) and Cy5 (Internal Control (IC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 3

OPEN AND ROTOR-GENE FORMAT WITH EXTRACTION CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 1 x 8-well strips, low profile	VS-MEP101LE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 1 x 8-well strips, high profile	VS-MEP101HE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 6 x 8-well strips, low profile	VS-MEP106LE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 6 x 8-well strips, high profile	VS-MEP106HE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 12 x 8-well strips, low profile	VS-MEP112LE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 12 x 8-well strips, high profile	VS-MEP112HE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 96-well plate, low profile	VS-MEP113LE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 96-well plate, high profile	VS-MEP113HE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 2 x 4-well strips, Rotor-Gene®	VS-MEP101E
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 9 x 4-well strips, Rotor-Gene®	VS-MEP136E
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit 18 x 4-well strips, Rotor-Gene®	VS-MEP172E

Table A3. 1. References.

A3.1 Principle of the procedure

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer, polymerase and retrotranscriptase) in a stabilized format. The assay can use an Extraction Control (EC) which can be introduced into each sample at the lysis buffer stage of the extraction process. This control can be used to monitor the extraction process and/or discard the inhibition of the polymerase activity.

Target	Channel	Gene
Mumps virus	FAM	F gene
Enterovirus	HEX, VIC or JOE *	5'UTR gene
Parechovirus	ROX	5'UTR gene
Extraction control (EC)	Cy5	-

Table A3. 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult website www.ceratest.es.

A3.2 Reagents provided

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit includes the following materials and reagents detailed in Tables A3.3, A3.4 and A3.5. Based on the commercial presentation and the Real Time PCR platform used, the stabilized PCR reaction mix could be placed inside different wells and could be marketed on multiple formats. Table A3.3 includes materials and reagents to be used with 8-well strips compatible devices. Table

A3.4 includes materials and reagents to be used with 96-well plate compatible devices. Table A3.5 includes materials and reagents for use with Qiagen/Corbett Rotor-Gene® instruments for 4-well strips. (Consult the thermocycler compatibility on CerTest's website www.cerTEST.es).

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	1/6/12 x 8-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	1/6/12 x 8-cap strip

Table A3. 3. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref. VS-MEP101LE, VS-MEP101HE, VS-MEP106LE, VS-MEP106HE, VS-MEP112LE and VS-MEP112HE.

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	1 plate
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 x 8-cap strip

Table A3. 4. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref VS-MEP113LE and VS-MEP113HE.

Reagent/Material	Description	Colour	Amount
<i>Mumps, Enterovirus & Parechovirus</i> 4-well strips	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	Transparent	2/9/18 x 4-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Mumps, Enterovirus & Parechovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
4-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	2/9/18 X 4-cap strip

Table A3 5. Reagents and materials provided in VIASURE *Mumps, Enterovirus & Parechovirus* Real Time PCR Detection Kit with Ref. VS-MEP101E, VS-MEP136E and VS-MEP172E. For use with Qiagen/Corbett Rotor-Gene® instruments and compatible accessories with strips of 4 tubes 0.1 ml (72-Well Rotor and Locking Ring 72-Well Rotor).

A3.3 Test procedure

A3.3.1 Lyophilized extraction control

Recommendation is to open and manipulate the Extraction Control (EC) in pre-PCR laboratory area away from the lyophilized positive control. Reconstitute the lyophilized Extraction Control (green vial) by adding 500 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the Extraction Control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The Water RNase/DNase free vial must be utilized first to reconstitute the lyophilized Extraction Control in pre-PCR laboratory area, and subsequently, it can be used for reconstituting the lyophilized *Mumps*, *Enterovirus* & *Parechovirus* Positive Control in an area away from the other components.

A3.3.2 Lyophilized positive control

Mumps, *Enterovirus* & *Parechovirus* Positive Control contains high copies of the template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A3.3.3 PCR protocol

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run for each assay. Peel off protective aluminium seal from plates or strips.

- 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

- 2) Adding samples and controls.

Add 5 µL of Negative Control (violet vial) in the reserved wells for negative control.

Add 5 µL of RNA sample in different wells.

Add 5 µL of reconstituted *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) in the reserved wells for positive control.

If the EC is only used as PCR inhibition control, add 1 µL of the EC (green vial) to the negative control/sample/positive control wells.

It is recommended to briefly centrifuge the 8-well strips or 96-well plate, or gently tap each strip onto a hard surface to ensure that all the liquids are at the bottom of the tubes (for Qiagen/Corbett Rotor-Gene® kit).

Load the plate or the strips in the thermocycler.

3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A3. 6. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (Mumps virus), ROX (Parechovirus), HEX, JOE or VIC (Enterovirus) and Cy5 (Extraction Control (EC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 4

TUBE FORMAT WITH EXTRACTION CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit, 4 tubes x 24 reactions	VS-MEP196TE

Table A4. 1. References.

A4.1 Principle of the procedure

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit contains in each Reaction-Mix tube all the components necessary for 24 real time PCR reactions (specific primers/probes, dNTPs, buffer, polymerase and retrotranscriptase) in a stabilized format. The assay can use an Extraction Control (EC) which can be introduced into each sample at the lysis buffer stage of the extraction process. This control can be used to monitor the extraction process and/or discard the inhibition of the polymerase activity.

Target	Channel	Gene
Mumps virus	FAM	F gene
Enterovirus	HEX, VIC or JOE *	5'UTR gene
Parechovirus	ROX	5'UTR gene
Extraction control (EC)	Cy5	-

Table A4. 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult website www.cerest.es.

A4.2 Reagents provided

VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit includes the following materials and reagents detailed in Table A4.3.

Reagent/Material	Description	Colour	Amount
Mumps, Enterovirus & Parechovirus Reaction-Mix tube	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	4 vials
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
Mumps, Enterovirus & Parechovirus Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL

Table A4. 3. Reagents and materials provided in VIASURE Mumps, Enterovirus & Parechovirus Real Time PCR Detection Kit with Ref. VS-MEP196TE.

A4.3 Test procedure

A4.3.1 Lyophilized extraction control

Recommendation is to open and manipulate the Extraction Control (EC) in pre-PCR laboratory area away from the lyophilized positive control. Reconstitute the lyophilized Extraction Control (green vial) adding 500 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the Extraction Control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The Water RNase/DNase free vial must be utilized first to reconstitute the lyophilized Extraction Control in pre-PCR laboratory area, and subsequently, it can be used for reconstituting the lyophilized *Mumps*, *Enterovirus* & *Parechovirus* Positive Control in an area away from the other components.

A4.3.2 Lyophilized positive control

Mumps, *Enterovirus* & *Parechovirus* Positive Control contains high copies of the template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A4.3.3 Lyophilized reaction mix tube

Determine the number of required reactions including samples and controls (one positive and negative control must be included in each run). Obtain the correct number of lyophilized Reaction-Mix vials (24-reactions each one) for testing.

Recommendation is to open and manipulate the *Mumps*, *Enterovirus* & *Parechovirus* Reaction-Mix tube in pre-PCR laboratory area. Open lyophilized Reaction-mix tube (white vial) carefully to avoid disruption of the pellet and add 390 µL of Rehydration Buffer (blue vial) supplied. Mix gently by pipetting up and down. Spin down briefly to remove bubbles generated during mixing.

Once the Reaction-Mix tube has been re-suspended, return unused reagents to the appropriate storage conditions at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The volume of the rehydrated Reaction-Mix is sufficient for 24 reactions. The rehydrated Reaction-Mix may be kept at 25°C±5°C or 2-8°C for up to 4-hours (see Transport and storage conditions section for additional storage options).

A4.3.4 PCR protocol

- 1) Adding rehydrated Reaction-Mix to the number of required wells.

Add 15 µL of rehydrated *Mumps*, *Enterovirus* & *Parechovirus* Reaction-Mix (white vial) into each tube.

2) Adding samples and controls.

Add 5 µL of Negative Control (violet vial) in the reserved wells for negative control.

Add 5 µL of RNA sample in different wells.

Add 5 µL of reconstituted *Mumps*, *Enterovirus* & *Parechovirus* Positive Control (red vial) in the reserved wells for positive control.

If the EC is only used as PCR inhibition control, add 1 µL of the EC (green vial) to the negative control/sample/positive control wells.

Close the wells with the caps provided. Centrifuge briefly.

Load the plate, the strips or tube in the thermocycler.

3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A4. 4. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Mumps* virus), ROX (*Parechovirus*), HEX, JOE or VIC (*Enterovirus*) and Cy5 (Extraction Control (EC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

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Modification rights reserved. All rights reserved. © CerTest Biotec, S.L.
All other trademarks that may appear in this package insert are the property of their respective owners.