VIASURE

Real Time PCR Detection Kit

Enterobacter,
A. baumannii & E. coli

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EN For download IFUS from other languages, please enter in certest.es/viasure/labeling. Once you be there, please following the instructions for access to the language that you need. If you need additional information, please contact: viasure@certest.es.

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Note: The user should notify the manufacturer and the competent authority of the Member State in which he is established as a user and/or patient of any serious incident related to the product.

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Content

1.	Intended use.....	8
2.	Summary and Explanation	8
3.	Principle of the procedure	9
4.	Reagents provided	9
5.	Reagents and equipment to be supplied by the user	9
6.	Transport and storage conditions.....	10
7.	Precautions for users	10
8.	Test procedure	12
8.1.	Specimen collection, transport and storage	12
8.2.	DNA extraction.....	12
9.	Result interpretation	13
9.1.	References with internal control (references in Annex 1 and 2).....	13
9.2.	References with extraction control (references in Annex 3 and 4)	15
10.	Limitations of the test	17
11.	Quality control.....	18
12.	Performance characteristics.....	18
12.1.	Clinical sensitivity and specificity.....	18
12.2.	Analytical sensitivity	19
12.3.	Analytical specificity	20
12.4.	Analytical reactivity	22
	ANNEX 1	22
A1.1	Principle of the procedure.....	23
A1.2	Reagents provided.....	23
A1.3	Test procedure	25
A1.3.1	Lyophilized positive control.....	25
A1.3.2	PCR protocol.....	25
	ANNEX 2	26
A2.1	Principle of the procedure.....	26
A2.2	Reagents provided.....	26
A2.3	Test procedure	26
A2.3.1	Lyophilized positive control.....	26
A2.3.2	Lyophilized reaction mix tube.....	27
A2.3.3	PCR protocol.....	27
	ANNEX 3	29
A3.1	Principle of the procedure.....	29
A3.2	Reagents provided.....	29
A3.3	Test procedure	31
A3.3.1	Lyophilized extraction control	31
A3.3.2	Lyophilized positive control.....	31
A3.3.3	PCR protocol.....	31
	ANNEX 4	33
A4.1	Principle of the procedure.....	33
A4.2	Reagents provided.....	33

A4.3 Test procedure	34
A4.3.1 Lyophilized extraction control	34
A4.3.2 Lyophilized positive control	34
A4.3.3 Lyophilized reaction mix tube.....	34
A4.3.4 PCR protocol.....	35

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ENGLISH

1. Intended use

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit is a real time PCR assay designed for the qualitative detection and differentiation of DNA from species of *Enterobacter*, *Acinetobacter baumannii* and/or *Escherichia coli*, from blood culture and swabs samples such as biopsy, gluteal swabs, ulcer swabs, wound swabs, tongue swab, joint fluid swab, oral swabs, hair swabs, perineal swab, urethral swab, abscess swabs and ostomy swabs from patients with suspect of bacterial infection and/or multi-resistant infection or colonization by their healthcare professional (HCP), as well as from swab specimens for epidemiological control such as nasal, pharyngeal, rectal swabs, axillary swabs, inguinal swabs, epithelial swabs, colostomy swabs, stoma swab, bronchoalveolar aspirates (BAS), bronchoalveolar lavages (BAL) and sputum specimens. This test is intended to be used as an aid in the diagnosis of *Enterobacter*, *A. baumannii* and/or *E. coli* infection, in combination with clinical and epidemiological risk factors. DNA is extracted from clinical samples, amplified using real time PCR, and detected using fluorescent reporter dye probes specific for *Enterobacter*, *A. baumannii* and *E. coli*.

2. Summary and Explanation

Antimicrobial resistance (AMR) has emerged as a substantial and triggering phenomenon with increasing costs for healthcare systems worldwide. In recent years it has been related to significant morbidity, mortality, and increased cost due to both prolonged length of hospitalization and treatment. ESKAPE organisms (*Enterococcus faecium*, *Staphylococcus aureus*, *Klebsiella pneumoniae*, *Acinetobacter baumannii*, *Pseudomonas aeruginosa*, and *Enterobacter spp.*), pose a global threat to human health and a therapeutic challenge due to emerging and constantly increasing resistance.

Enterobacter spp. are rod-shaped, gram-negative bacteria from *Enterobacteriaceae* family. Twenty-two species belong to *Enterobacter* genus and they are described in the environment and have been reported as opportunistic pathogens in plants, animals, and humans. *E. cloacae* is the most frequent nosocomial pathogen among *Enterobacter* species, being able to cause urinary tract infections, bacteremia or lower respiratory tract infections and wound infections. The pathogenicity/virulence of this bacterium remains unclear due to the lack of studies in this field. In contrast, its resistance against antibacterial agents has been extensively studied. Frequently, the acquisition of *Enterobacter spp.* concerned multidrug-resistant (MDR) strains.

Acinetobacter baumannii is a Gram-negative, multidrug-resistant (MDR) pathogen associated with severe infections, most typically nosocomial pneumonia and bloodstream infections in critically ill patients. Moreover, *A. baumannii* has become an increasingly relevant human pathogen due to the rise in the number of infections that causes and the emergence of multidrug-resistant strains. *A. baumannii* has developed a broad spectrum of antimicrobial resistance, associated with a higher mortality rate among infected patients compared with other *non-baumannii* species.

Escherichia coli is a microorganism responsible for human and animal infections, whose genetic variability allows it to grow in diverse ecological niches. It is recognized as one of the most frequent bacterial cause of infections, food-borne diarrheal disease, and extraintestinal infections. Pathogenic variants of *E. coli* (pathovars or pathotypes) cause much morbidity and mortality worldwide, because they have low infectious doses and are transmitted through ubiquitous mediums, including food and water. Although not formally recognized as part of

the ESKAPE group of pathogens, AMR *E. coli* is identified as a major cause of bloodstream and urinary tract infection (UTI) in both community and health care settings globally.

The most common diagnostic methods are culture-based. However, conventional culture method is time-consuming. Molecular diagnostic based on qPCR is proposed as a future alternative for the detection of these pathogens in a more sensitive and specific way.

3. Principle of the procedure

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit is designed for the qualitative detection and differentiation of DNA from *Enterobacter*, *A. baumannii* and/or *E. coli*, from blood culture and swabs samples. After DNA isolation, the identification of *Enterobacter*, *A. baumannii* and *E. coli* is performed by the amplification of a conserved region of the *Xx* and *Omp* genes of *Enterobacter*, *bap* gene of *A. baumannii*, and the *16S rRNA* gene of *E. coli*, using specific primers and a fluorescent-labelled probes.

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit is based on 5' exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal, which is proportional to the quantity of target template. This fluorescence could be measured on Real Time PCR platforms.

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer and polymerase) in a stabilized format, as well as an **internal control** to verify the correct functioning of the amplification mix.

4. Reagents provided

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit includes the materials and reagents detailed in Annex 1 for open and rotor-gene format with internal control products, Annex 2 for tube format with internal control products, Annex 3 for open and rotor-gene format with extraction control products, and Annex 4 for tube format with extraction control products.

5. Reagents and equipment to be supplied by the user

The following list includes the materials that are required for use but not included in the VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler).
- Real Time PCR compatible plastic consumables (i.e. individual tubes, well-strips and/or microplates). Only for Tube format (Annex 2 and 4).
- DNA extraction kit.
- Collection and transport system.
- Laboratory freezers: - 30°C to - 10°C and/or ≤ -70°C.
- Centrifuge for 1.5 mL tubes and PCR-well strips or 96-well plate (if available).
- Vortex.
- Micropipettes (0.5-20 µL, 20-200 µL).

- Filter tips.
- Powder-free disposable gloves.
- Loading block (for use with Qiagen/Corbett Rotor-Gene® instruments). Only for Rotor Gene format (Annex 1 and 3).

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit has been tested on the following equipments: 7500 Fast Real-Time PCR Systems (Applied Biosystems), CFX96 Touch™ Real-Time PCR Detection System (Bio-Rad), AriaMx Real-Time PCR System (Agilent Technologies), DTprime Real-time Detection Thermal Cycler (DNA-Technology), DTlite Real-Time PCR System (DNA-Technology), Rotor-Gene® Q (Qiagen), Cobas z480 Analyzer (Roche Molecular Diagnostics), and VIASURE V-Lab96 Cycler (CerTest Biotec S.L.). To check thermocycler compatibility and most common detection channels consult website www.certest.es.

Optical measurement parameters of some thermocyclers must be adjusted to be suitable for operation with VIASURE Real Time PCR Detection Kits. This assay has been validated with the following set exposition values:

- DTprime Real-time Detection Thermal Cycler (DNA-Technology): FAM channel -500*, HEX channel – 1000, ROX channel – 1000 and Cy5 channel - 1000.
- DTlite Real-Time PCR System (DNA-Technology): FAM channel - 500, HEX channel - 500, ROX channel – 500 and Cy5 channel - 500.

*If the result in channel FAM is not as expected, there are no amplifications or high background noise is observed, please lower the exposure values indicated above to 150.

6. Transport and storage conditions

- The kits can be shipped and stored at 2-40°C until the expiration date which is stated on the label.
- Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles. Positive control has been validated as still being stable after 6 freeze-thaw cycles.
- Keep components away from light.
- For Tube format kits: Once the *Enterobacter*, *A. baumannii* & *E. coli* Reaction-Mix tube has been reconstituted, it may be kept at 25°C±5°C or 2°C to 8°C for up to 4 hours. For a longer period of time, it is recommended store at -20°C and to separate in aliquots to minimize freeze and thaw cycles (up to 6 times).

7. Precautions for users

- The product is intended for use by qualified and trained clinical laboratory personnel specifically instructed and trained in the techniques of real-time PCR and *in vitro* diagnostic procedures (including training on the Real Time PCR instrument (thermocycler) and Nucleic acid extraction system).
- For *in vitro* diagnostic use.
- Do not use expired reagents and/or materials.
- Do not use the kit if the label that seals the outer box is broken.
- Do not use reagents if the protective box is open or broken upon arrival.
- Do not use reagents if the protective pouches are open or broken upon arrival.

- Do not use reagents if desiccant is not present or broken inside reagent pouches.
- Do not remove desiccant from reagent pouches once is open.
- Close protective pouches of reagents promptly with the zip seal after each use (for references: VS-EAC113L, VS-EAC113H, VS-EAC136, VS-EAC172, VS-EAC113LE, VS-EAC113HE, VS-EAC136E and VS-EAC172E). Remove any air excess in the pouches prior to closing.
- Do not use reagents if the foil has been broken or damaged.
- Do not mix reagents from different pouches and / or kits and / or lots and / or another supplier.
- Protect reagents against from humidity. Prolonged exposure to humidity may affect product performance.
- For references VS-EAC101, VS-EAC136, VS-EAC172, VS-EAC101E, VS-EAC136E and VS-EAC172E (compatible for use with Qiagen/Corbett Rotor-Gene® instruments) use the loading block to pipette reagents and samples into each tube and to help with fitting caps properly and avoid cross contamination.
- An appearance of the reaction mixture in stabilized format, normally found at the bottom of the tube, different from the usual one (without conical shape, inhomogeneous, smaller/larger in size and/or colour different from whitish) does not alter the functionality of the test.
- Design a unidirectional workflow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed. Use separate areas for the preparation of patient samples and controls to prevent false positive results.
- In cases where other PCR tests are conducted in the same general area of the laboratory, care must be taken to ensure that the VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit and any additional reagents or equipments required for testing are not contaminated. Always avoid microbial and ribonuclease (RNase)/deoxyribonuclease (DNase) contamination of reagents. The use of sterile RNase/DNase-free disposable aerosol resistant or positive displacement pipette tips is recommended. Use a new tip for each specimen. Gloves must be changed before manipulating reagents.
- Follow Good Laboratory Practices. Wear protective clothing, use disposable gloves, goggles and mask. Do not eat, drink, smoke or apply cosmetic products in the working area. Wash your hands after finishing the test.
- Specimens must be treated as potentially infectious and/or biohazardous, as well as all the reagents and materials that have been exposed to the samples and they must be handled according to the national safety regulations. Take necessary precautions during the collection, transport, storage, handling, and disposal of samples.
- Samples and reagents must be handled in a biological safety cabinet. Use personal protective equipment (PPE) consistent with current guidelines for the handling of potentially infectious samples. Dispose of waste in compliance with local and state regulations.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.
- In accordance with Regulation (EC) No 1907/2006 (REACH), VIASURE Real Time PCR Detection Kits do not require Material Safety Data Sheets on account of their classification as non-hazardous to health and the environment, because they do not contain substances and/or mixtures which meet the hazard classification criteria available in Regulation (EC) No 1272/2008 (CLP), or which are in concentrations higher than the value established in the mentioned regulation for their declaration.
- Consult each Real Time PCR instrument's reference manual for additional warnings, precautions and procedures.

8. Test procedure

Please see Annex 1 for open and rotor-gene format with internal control products Test Procedure, Annex 2 for Tube format with internal control products Test Procedure, Annex 3 for open and rotor-gene format with extraction control products Test Procedure and Annex 4 for Tube format with extraction control products Test Procedure.

8.1. Specimen collection, transport and storage

The VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit has been tested in blood culture samples collected in aerobic vials BD BACTEC™ Plus Aerobic/F and in anaerobic vials BD BACTEC™ Lytic/10 Anaerobic/F (Becton Dickinson), as well as in swab samples (biopsy, gluteal, ulcer, wound, tongue, joint fluid, oral, hair, perineal, urethral, abscess, ostomy, nasal, pharyngeal, rectal, axillar, inguinal, epithelial, colostomy, stoma, BAS, BAL, sputum) collected using a sterile breakable polystyrene swab and placed into AMIES transport medium (Deltalab). Other types of samples must be validated by the user.

Collection, storage and transport of specimens should be maintained per the conditions validated by the user. Overall, all specimens should be collected and labelled appropriately in clean containers. After collection, specimens should be placed in a biohazard bag and should be transported and processed as soon as possible to guarantee the quality of the test. The specimens should be transported at Room Temperature (RT) for up to 2 hours, following the local and national regulations for the transport of pathogen material. For long term transport (more than 2 hours), it is recommend shipping at -20°C or lower if preservation medium permits. It is recommended to use fresh specimens for the test. The samples can be stored at -80°C for conservation, **if preservation medium permits**. We recommend following appropriate laboratory guidelines and/or manufacturer's recommendations for proper preservation of samples in the selected collection and transport medium. Repeated freeze-thaw cycles should be avoided in order to prevent degradation of the sample and nucleic acids.

The clinical specimens must be collected, transported and stored according to appropriate laboratory guidelines. For details, refer to the CDC guideline (Specimen collection guidelines. Website <https://www.cdc.gov/urdo/downloads/SpecCollectionGuidelines.pdf>), the IDSA guideline (Miller, J. M., Binnicker, M. J., Campbell, S., ... & Pritt, B. S. (2018). A guide to utilization of the microbiology laboratory for diagnosis of infectious diseases: 2018 update by the Infectious Diseases Society of America and the American Society for Microbiology. *Clinical Infectious Diseases*, 67(6), e1-e94), and Garcia-Lechuz Moya, J.M., Gonzalez Lopez, J.J., Orta Mira, N., Sanchez Romero, M.I. (2017). Recogida, transporte y procesamiento general de las muestras en el Laboratorio de Microbiología. 2017. 1b. Sanchez Romero, M.I., (coordinadora). Procedimientos en Microbiología Clínica. Cercenado Mansilla, E., Canton Moreno, R., (editores). *Sociedad Española de Enfermedades Infecciosas y Microbiología Clínica (SEIMC)*.

8.2. DNA extraction

Perform the sample preparation according to the recommendations appearing in the instructions for use of the extraction kit used.

Due to VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit is also available with extraction control (references include in Annexes 3 and 4), if the Extraction Control (EC) is used to monitor nucleic acid isolation and as PCR inhibition control, add 5µl of the reconstituted EC to the specimen and/or lysis buffer mixture (clinical specimen, as well as positive control and/or negative control). Close each tube and vortex for 10 seconds. If the Extraction Control is used only as a PCR inhibition control, 1µl of the reconstituted EC should be added to the reconstituted Reaction-Mix.

For DNA extraction from clinical samples, you can use your manual or automatic routine optimized system, or any commercially available DNA extraction kit and follow the manufacturer's instructions. The following extraction kits have been validated:

- MagDEA Dx SV kit, using the magLEAD® 12gC instrument (Precision System Science Co.).
- Invisorb® Spin Universal Kit (Invitex).

9. Result interpretation

9.1. References with internal control (references in Annex 1 and 2)

All the results of the test should be evaluated by a healthcare professional in the context of medical history, clinical symptoms, and other diagnostic tests. Check Internal Control (IC) signal to verify the correct functioning of the amplification mix. The analysis of the controls and samples is done by the software of the used real time PCR equipment itself according to manufacturer's instructions.

It is recommended to set the threshold values for each channel (target) independently by the end-user. Use the Positive Control amplification curve as a starting point during the run validation (before than interpretation of patient sample results), in order to ensure that thresholds fall within the exponential phase of the fluorescence curves and above any background signal. The threshold value for different instruments may vary due to different signal intensities.

The use of positive and negative controls in each run validates the reaction by checking the absence of signal in the negative control well and the presence of signal for *Enterobacter*, *A. baumannii* and *E. coli* in the positive control well.

For a valid diagnostic test run, the following control conditions must be met:

Controls	<i>Enterobacter</i> (FAM) ¹	<i>A. baumannii</i> (ROX) ¹	<i>E. coli</i> (Cy5) ¹	Internal Control (HEX) ²	Interpretation of Controls
Positive Control (PC)	≤40	≤40	≤40	≤40	Valid
Negative Control (NC)	> 35 or no signal	> 35 or no signal	> 30 or no signal	≤40	Valid

Table 1. Expected Performance of Controls. Ct values no signal = no amplification curve.

1. In cases where either or both control assays have failed (an amplification signal is observed in the negative control and/or signals absence in the positive control well for any target channel), all results are reported as 'Invalid', and retesting is required.

2. The Internal Control (IC) should show an amplification signal (Ct ≤40) in PC and NC control wells.

Assessment of clinical samples test results should be performed after the positive and negative controls have been examined and determined to be valid and acceptable. If one or more controls are not valid, the patient results cannot be interpreted.

For interpretation of individual patient sample results, use the following table, read and analyze the results:

<i>Enterobacter</i> (FAM)	<i>A. baumannii</i> (ROX)	<i>E. coli</i> (Cy5)	Internal Control (HEX)	Interpretation for patients' individual samples	
≤35	> 35 or no signal	> 30 or no signal	≤40 or no signal ¹	Valid	<i>Enterobacter</i> DNA Detected
> 35 or no signal	≤35	> 30 or no signal	≤40 or no signal ¹	Valid	<i>A. baumannii</i> DNA Detected
> 35 or no signal	> 35 or no signal	≤30	≤40 or no signal ¹	Valid	<i>E. coli</i> DNA Detected
≤35	≤35	> 30 or no signal	≤40 or no signal ¹	Valid	<i>Enterobacter</i> and <i>A. baumannii</i> DNA Detected
≤35	> 35 or no signal	≤30	≤40 or no signal ¹	Valid	<i>Enterobacter</i> and <i>E. coli</i> DNA Detected
> 35 or no signal	≤35	≤30	≤40 or no signal ¹	Valid	<i>A. baumannii</i> and <i>E. coli</i> DNA Detected
≤35	≤35	≤30	≤40 or no signal ¹	Valid	All targets' DNA Detected
> 35 or no signal	> 35 or no signal	> 30 or no signal	≤35 ²	Valid	Targets not Detected
> 35 or no signal	> 35 or no signal	> 30 or no signal	> 35 or no signal ²	Invalid	Test Failure – Repeat Testing ²

Table 2. Interpretation of individual patient sample results. Ct values no signal = no amplification curve.


¹ The Internal Control (IC) shows or not an amplification signal (Ct ≤40 or no signal). Sometimes, its detection is not necessary because a high copy number of the target can cause preferential amplification of target-specific nucleic acids.

² In the case of *Enterobacter*, *A. baumannii* and *E. coli* target genes negative, IC must show an amplification signal with Ct ≤ 35. If there is an absence of signal or Ct value > 35 of the Internal Control, the result is considered as 'Invalid', and retesting is required. It is recommended to repeat the qPCR diluting the DNA sample 1:10 and/or 1:100, or re-extract and retest to check for possible failure in the extraction procedure and/or inhibition issues.

In case of a continued ambiguous result, it is recommended to review the instructions for use, the extraction process used by the user, to verify the correct performance of each qPCR steps and review the parameters, and to check the sigmoid shape of the curve and the intensity of fluorescence. It is also recommended to repeat the assay, preferably in duplicate. Depending on the available material:

- repeat qPCR with the same isolated DNA sample, or
- re-extract and retest another aliquot of the same specimen or,
- obtain a new specimen and retest.

Figure 1. Correct run of negative and positive control run on the CFX96™ Real-Time PCR Detection System (Bio-Rad).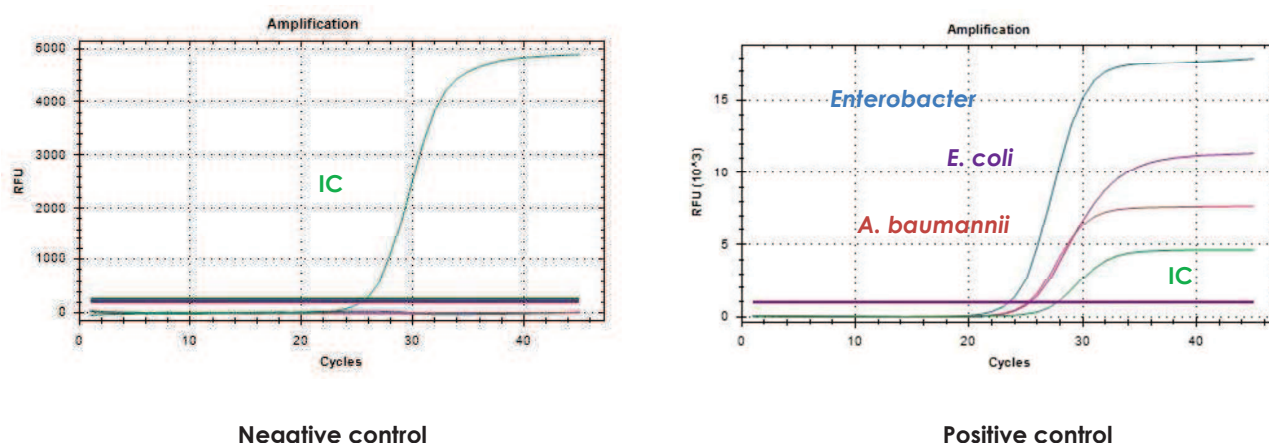

9.2. References with extraction control (references in Annex 3 and 4)

All the results of the test should be evaluated by a healthcare professional in the context of medical history, clinical symptoms, and other diagnostic tests. Check Extraction Control (EC) signal to verify the correct functioning of the amplification mix. The analysis of the controls and samples is done by the software of the used real time PCR equipment itself according to manufacturer's instructions.

It is recommended to set the threshold values for each channel (target) independently by the end-user. Use the Positive Control amplification curve as a starting point during the run validation (before than interpretation of patient sample results), in order to ensure that thresholds fall within the exponential phase of the fluorescence curves and above any background signal. The threshold value for different instruments may vary due to different signal intensities.

The use of positive and negative controls in each run validates the reaction by checking the absence of signal in the negative control well and the presence of signal for *Enterobacter*, *A. baumannii* and *E. coli* in the positive control well.

For a valid diagnostic test run, the following control conditions must be met:

Controls	<i>Enterobacter</i> (FAM) ¹	<i>A. baumannii</i> (ROX) ¹	<i>E. coli</i> (Cy5) ¹	Extraction Control (HEX) ²	Interpretation of Controls
Positive Control (PC)	≤40	≤40	≤40	≤40	Valid
Negative Control (NC)	> 35 or no signal	> 35 or no signal	> 30 or no signal	≤40	Valid

Table 3. Expected Performance of Controls. Ct values no signal = no amplification curve.

1 In cases where either or both of the control assays have failed (an amplification signal is observed in the negative control and/or signals absence in the positive control well for any target channel), all results are reported as 'Invalid', and retesting is required.

2. The Extraction Control (EC) should show an amplification signal (Ct ≤40) in PC and NC control wells.

Assessment of clinical samples test results should be performed after the positive and negative controls have been examined and determined to be valid and acceptable. If one or more controls are not valid, the patient results cannot be interpreted.

For interpretation of individual patient sample results, use the following table, read and analyze the results:

<i>Enterobacter</i> (FAM)	<i>A. baumannii</i> (ROX)	<i>E. coli</i> (Cy5)	Extraction Control (HEX)	Interpretation for patients' individual samples	
≤35	> 35 or no signal	> 30 or no signal	≤40 or no signal ¹	Valid	<i>Enterobacter</i> DNA Detected
> 35 or no signal	≤35	> 30 or no signal	≤40 or no signal ¹	Valid	<i>A. baumannii</i> DNA Detected
> 35 or no signal	> 35 or no signal	≤30	≤40 or no signal ¹	Valid	<i>E. coli</i> DNA Detected
≤35	≤35	> 30 or no signal	≤40 or no signal ¹	Valid	<i>Enterobacter</i> and <i>A. baumannii</i> DNA Detected
≤35	> 35 or no signal	≤30	≤40 or no signal ¹	Valid	<i>Enterobacter</i> and <i>E. coli</i> DNA Detected
> 35 or no signal	≤35	≤30	≤40 or no signal ¹	Valid	<i>A. baumannii</i> and <i>E. coli</i> DNA Detected
≤35	≤35	≤30	≤40 or no signal ¹	Valid	All targets' DNA Detected
> 35 or no signal	> 35 or no signal	> 30 or no signal	≤35 ²	Valid	Targets not Detected
> 35 or no signal	> 35 or no signal	> 30 or no signal	> 35 or no signal ²	Invalid	Test Failure – Repeat Testing ²

Table 4. Interpretation of individual patient sample results. Ct values. no signal = no amplification curve.


¹ The Extraction Control (EC) shows or not an amplification signal (Ct ≤40 or no signal). Sometimes, its detection is not necessary because a high copy number of the target can cause preferential amplification of target-specific nucleic acids. Differences can be observed in the values of Ct in the extraction control between the controls and the clinical samples, due to the extraction process.

² In the case of *Enterobacter*, *A. baumannii* and *E. coli* target genes negative, EC must show an amplification signal with Ct ≤ 35. If there is an absence of signal or Ct value > 35 of the Extraction Control, the result is considered as 'Invalid', and retesting is required. It is recommended to repeat the qPCR diluting the DNA sample 1:10 and/or 1:100, or re-extract and retest to check for possible failure in the extraction procedure and/or inhibition issues.

In case of a continued ambiguous result, it is recommended to review the instructions for use, the extraction process used by the user, to verify the correct performance of each qPCR steps and review the parameters, and to check the sigmoid shape of the curve and the intensity of fluorescence. It is also recommended to repeat the assay, preferably in duplicate. Depending on the available material:

- a) repeat qPCR with the same isolated DNA sample, or
- b) re-extract and retest another aliquot of the same specimen or,
- c) obtain a new specimen and retest.

Figure 2. Correct run of negative and positive control run on the CFX96™ Real-Time PCR Detection System (Bio-Rad).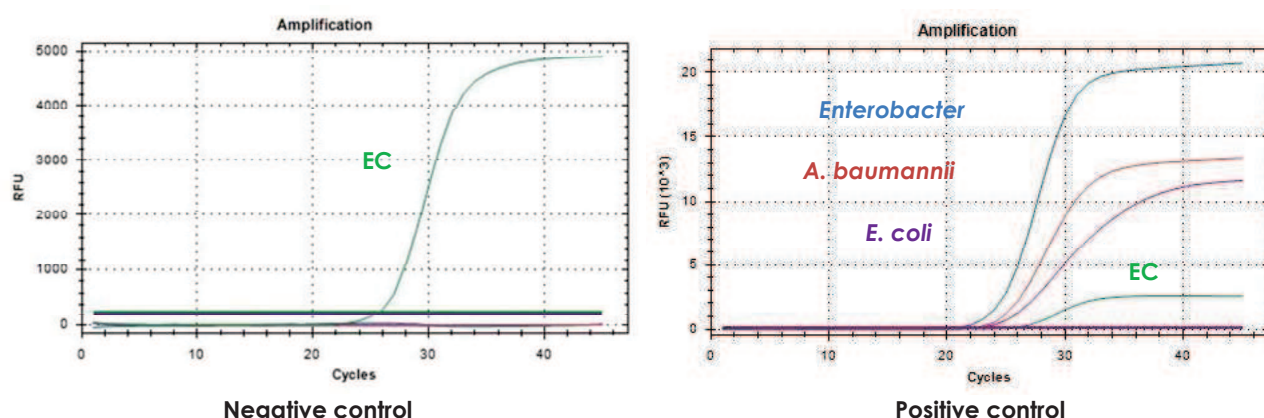

10. Limitations of the test

- The results of the test should be evaluated by a health care professional in the context of medical history, clinical symptoms and other diagnostic tests.
- Although this assay can be used with other types of samples, it has been validated only with DNA extracted from blood culture and swabs samples.
- The quality of the test depends on the quality of the sample; nucleic acid must be properly extracted from clinical samples.
- This test is a qualitative test and does not provide quantitative values or indicate the number of organisms present.
- Extremely low levels of target below the limit of detection might be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by *Enterobacter*, *A. baumannii* and/or *E. coli*, either by samples containing high concentrations of target DNA or contamination due to PCR products from previous reactions.
- There is a possibility of false positive results due to cross-contamination between Extraction Control and *Enterobacter*, *A. baumannii* & *E. coli* Positive Control, which contains high copies template, during their reconstitution by adding of Water RNase/DNase free (white vial). Each procedure must take place in established order and in a separate laboratory areas.
- False Negative results may arise from several factors and their combinations, including:
 - Improper specimens' collection, transport, storage, and/or handling methods.
 - Improper processing procedures (including DNA extraction).
 - Degradation of the DNA during sample shipping/storage and/or processing.
 - Mutations or polymorphisms in primer or probe binding regions may affect detection of *Enterobacter*, *A. baumannii* and/or *E. coli*.
 - A bacterial load in the specimen below the limit of detection for the assay.
 - The presence of qPCR inhibitors or other types of interfering substances. The impacts of vaccines, antiviral therapeutics, antibiotics, chemotherapeutics, or immunosuppressant drugs, used to prevent *Enterobacter*, *A. baumannii* and/or *E. coli* infections or used during the treatment of the infection have not been evaluated.
 - Failure to follow instructions for use and the assay procedure.

- A positive test result does not necessarily indicate the presence of viable bacteria and does not imply that these bacteria are infectious or are the causative agents for clinical symptoms. However, a positive result is indicative of the presence of *Enterobacter*, *A. baumannii* and/or *E. coli* target sequences.
- Negative results do not preclude *Enterobacter*, *A. baumannii* and/or *E. coli* infection and should not be used as the sole basis for treatment or other patient management decisions. Optimum specimen types and timing for peak bacteria levels during infections caused by *Enterobacter*, *A. baumannii* and/or *E. coli* have not been determined. The collection of multiple specimens (types and time points) from the same patient may be necessary to detect the target bacteria.
- If diagnostic tests for other bacterial infections and/or multi-resistant infection or colonization are negative and the patient's clinical presentation and epidemiological information suggest that *Enterobacter*, *A. baumannii* and/or *E. coli* infection is possible, then a false negative result should be considered, and a re-testing of the patient should be discussed.
- Fluorescence values may vary due to multiple factors such as: PCR equipment, extraction system, type of sample, previous treatment of the sample, etc... among others.

11. Quality control

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) or the Extraction control (EC) in each well confirms the correct performance of the technique.

12. Performance characteristics

12.1. Clinical sensitivity and specificity

The clinical performance of VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit was tested using clinical and/or epidemiological samples consisting of total-blood culture and swabs from patients suspected of bacterial infection and/or multi-resistant infection or colonization. In order to determine the clinical diagnostic accuracy, an evaluation has been conducted through collaboration with a national entity. A summary of the sites, sample type and workflow is included in the following table:

	Site	Sample type	Workflow	Target
1	CerTest Biotec S.L, in collaboration with the Microbiology Department of Hospital Royo Villanova (Zaragoza, Spain)	Blood culture	MagDEA® Dx SV Kit using the magLEAD® 12gC instrument (Precision System Science Co.) + DTprime Real-time Detection Thermal Cycler (DNA-Technology).	<i>Enterobacter</i>
				<i>A. baumannii</i>
				<i>E. coli</i>
2	CerTest Biotec S.L, in collaboration with the Microbiology Department of Hospital Royo Villanova (Zaragoza, Spain)	Swab samples*	MagDEA® Dx SV Kit using the magLEAD® 12gC instrument (Precision System Science Co.) + DTprime Real-time Detection Thermal Cycler (DNA-Technology).	<i>Enterobacter</i>
				<i>A. baumannii</i>
				<i>E. coli</i>

Table 5. Site, sample type, workflow and target.

* Swabs samples such as gluteal, ulcer, wound, tongue, joint fluid, oral, hair, perineal, urethral, abscess, axillary, inguinal, epithelial, ostomy, nasal, pharyngeal, rectal, stoma, biopsy, BAS, BAL, colostomy, and sputum.

True positive and negative values, false positive and negative values, sensitivity, specificity, PPV, NPV values for VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit were calculated in relation to each comparator assay as shown in the following table:

Site	Comparator assay	Target	Sample type	TP	TN	FP	FN	Sensitivity	Specificity	PPV	NPV
1	Culture + MALDI-TOF	<i>Enterobacter</i>	Blood culture	10	408	0	0	1 (0.69-1)	1 (0.99-1)	1 (0.72-1)	1 (0.99-1)
		<i>A. baumannii</i>		0	418	0	0	n.a.*	1 (0.98-1)	n.a.*	1 (0.99-1)
		<i>E. coli</i>		111	307	0	0	1 (0.96-1)	1 (0.98-1)	1 (0.96-1)	1 (0.98-1)
	Magicplex™ Sepsis ID 7 Real-time Detection Test / Magicplex™ Sepsis ID 4 Real-time Detection Test (Seegene Inc.)	<i>Enterobacter</i>	Blood culture	10	46	0	0	1 (0.69-1)	1 (0.92-1)	1 (0.69-1)	1 (0.92-1)
		<i>A. baumannii</i>		0	26	0	0	n.a.*	1 (0.86-1)	n.a.*	1 (0.86-1)
		<i>E. coli</i>		23	33	0	0	1 (0.85-1)	1 (0.89-1)	1 (0.85-1)	1 (0.89-1)
2	Culture + MALDI-TOF	<i>Enterobacter</i>	Swab samples	85	542	0	0	1 (0.95-1)	1 (0.99-1)	1 (0.95-1)	1 (0.99-1)
		<i>A. baumannii</i>		4	623	0	0	1 (0.39-1)	1 (0.99-1)	1 (0.39-1)	1 (0.99-1)
		<i>E. coli</i>		166	461	0	0	1 (0.97-1)	1 (0.99-1)	1 (0.97-1)	1 (0.99-1)

Table 6. True positive (TP) and true negative values (TN), false positive (FP) and false negative (FN) values, sensitivity, specificity, Predictive Positive Values (PPV) and Predictive Negative Values (NPV) for VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit.

* Sensitivity and PPV value not calculated as no positive samples for *A. baumannii* were found in total-blood culture samples.

Results show high agreement to detect *Enterobacter*, *A. baumannii* and/or *E. coli* using VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit.

12.2. Analytical sensitivity

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit showed a detection limit of 2×10^{-3} CFU/ μ L for *E. cloacae* subsp. *cloacae*, 2×10^{-3} CFU/ μ L for *A. baumannii*, and 5×10^{-3} CFU/ μ L for *E. coli*, with a positive rate of $\geq 95\%$, on blood samples, and 4×10^{-4} CFU/ μ L for *E. cloacae* subsp. *cloacae*, 10^{-3} CFU/ μ L for *A. baumannii* and 10^{-2} CFU/ μ L for *E. coli*, with a positive rate of $\geq 95\%$, on urine samples.

Figure 3. Dilution series of *Enterobacter* (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel FAM).

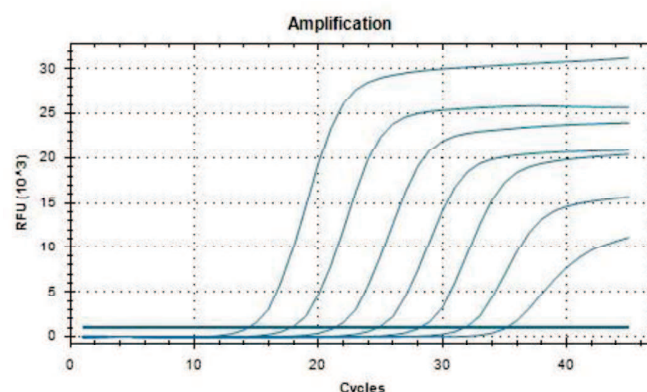
Figure 4. Dilution series of *A. baumannii* (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel ROX).


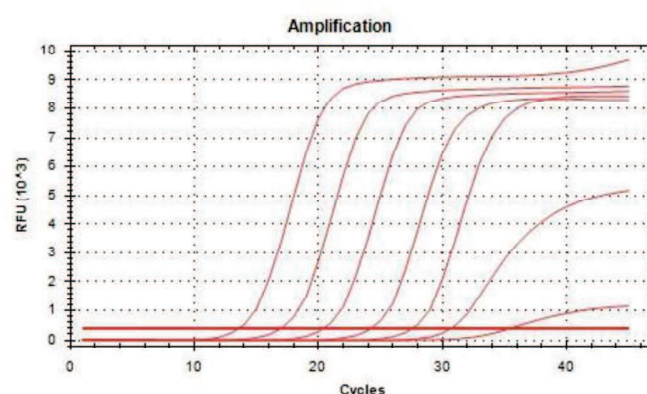
Figure 5. Dilution series of *E. coli* (10^7 - 10^1 copies/rxn) template run on the CFX96™ Real-Time PCR Detection System (Bio-Rad) (channel Cy5).

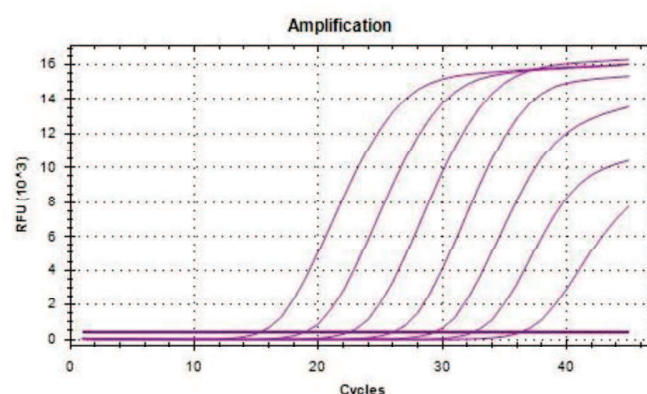12.3. Analytical specificity

The specificity of the VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit assay was confirmed by testing a panel of different microorganisms associated to gastrointestinal symptoms and/or blood transmission. No cross-reactivity of the VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit with any of the following microorganisms tested was observed, except for some samples with potential contamination by the target pathogen/s due to the matrix type. *Shigella* spp. and *E. coli* belong to the family *Enterobacteriaceae*, share many

common phenotypic characteristics and are closely genetic relatedness. In fact, the nucleotide similarity between *Shigella* and *E. coli* is 80% to 90%, which cause a challenging differentiation.

Cross-reactivity testing (wet testing)					
Adenovirus types 1-5, 8, 15, 31, 40 and 41 *	+/-	<i>Klebsiella oxytoca</i>	-	<i>Helicobacter hepaticus</i>	-
<i>Aeromonas caviae</i>	-	<i>Listeria monocytogenes</i> serovar 1/2c	-	<i>Streptococcus pneumoniae</i>	-
<i>Aeromonas hydrophila</i> subsp. <i>hydrophila</i>	-	Norovirus GI \$	-	<i>Borrelia miyamotoi</i>	-
<i>Arcobacter butzleri</i>	-	Norovirus GII (GII.4)	-	<i>Borrelia spielmanii</i>	-
Astrovirus genotypes I-VIII	-	<i>Proteus vulgaris</i>	-	<i>Coxiella burnetii</i> strain Nine Mile Q	-
<i>Bacteroides fragilis</i>	-	<i>Pseudomonas aeruginosa</i>	-	<i>Leptospira</i>	-
<i>Campylobacter coli</i>	-	<i>Salmonella bongori</i> serovar 66:z41:-	-	<i>Rickettsia conorii</i> strain Moroccan	-
<i>Campylobacter fetus</i> subsp. <i>fetus</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar enteritidis	-	<i>Theileria annulata</i>	-
<i>Campylobacter hyointestinalis</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar gallinarum	-	Tick-Borne encephalitis virus strain Neudorfi	-
<i>Campylobacter jejuni</i> subsp. <i>jejuni</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar paratyphi A and B	-	<i>Treponema phagedenis</i>	-
<i>Campylobacter lari</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar pullorum	-	Chikungunya S27 Petersfield	-
<i>Campylobacter upsaliensis</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar Typhimurium serotype 4,5,12:i:1,2.	-	Chikungunya Virus Martinique	-
<i>Candida albicans</i>	-	<i>Salmonella enterica</i> subsp. <i>enterica</i> serovar typhi	-	Chikungunya Virus F24	-
<i>Citrobacter freundii</i>	-	Sapovirus \$	+/-	Chikungunya Virus WHO IS (R91064)	-
<i>Clostridium difficile</i>	-	<i>Serratia liquefaciens</i>	-	Dengue 1 Hawaii A	-
<i>Clostridium difficile</i> 027 &	+/-	<i>Shigella dysenteriae</i> serotype 1 #	+/-	Dengue 2 New Guinea C	-
<i>Clostridium perfringens</i>	-	<i>Shigella flexneri</i> #	+/-	Dengue 3 H87	-
Coxsackievirus A24, A9 and B3	-	<i>Staphylococcus aureus</i> subsp. <i>aureus</i>	-	Dengue 4 H241	-
<i>Cryptosporidium parvum</i>	-	<i>Vibrio parahaemolyticus</i> serotype O1:K1	-	Japanese Encephalitis virus	-
<i>Cryptosporidium hominis</i>	-	<i>Yersinia enterocolitica</i> (O:3, O:9)	-	Japanese Encephalitis virus Nakayama	-
<i>Dientamoeba fragilis</i> / <i>Blastocystis hominis</i> \$	+/-	<i>Anaplasma marginale</i>	-	<i>Plasmodium falciparum</i> strain 3D7	-
Echovirus 30	-	<i>Bartonella henselae</i> strain Houston-1	-	Rift Valley Fever Virus AR21229	-
<i>Entamoeba dispar</i> / <i>Blastocystis hominis</i> \$	+/-	<i>Borrelia hermsii</i>	-	Rift Valley Fever Virus MP12	-
<i>Entamoeba histolytica</i> strain DS4-868	-	<i>Borrelia lusitanae</i>	-	St Louis Encephalitis Virus	-
<i>Enterococcus faecalis</i>	-	<i>Borrelia valaisiana</i>	-	<i>Trypanosoma cruzi</i>	-
Enterovirus 68 and 71	-	<i>Borrelia azfeli</i> strain P-Ko/1984	-	Usutu Virus	-
<i>Giardia intestinalis</i> strain WB clone C6	-	<i>Borrelia bavariensis</i>	-	West Nile Virus (NY99, Heja and Ug37)	-
<i>Helicobacter cinaedi</i>	-	<i>Borrelia bisetti</i>	-	Yellow Fever virus strain 17D	-
<i>Helicobacter heilmannii</i>	-	<i>Borrelia burgdorferi</i> strain IRS	-	Yellow Fever Virus French Neurotropic	-
<i>Helicobacter pylori</i> J99	-	<i>Borrelia burgdorferi sensu stricto</i> strain B31	-	Zika Virus FB-GWUH-2016	-
Human Rotavirus A	-	<i>Borrelia garinii</i> type strain	-	Zika Virus strains African, Asian PF13/251013-18, French Polynesian (11474/16 and 11468/16)	-
<i>Klebsiella pneumoniae</i>	-	<i>Borrelia japonica</i>	-		

Table 7. Reference pathogenic microorganisms used in this study performed at CerTest facility.

* It was overexpressed in culture and may carry DNA from *E. coli*. For Adenovirus Type 2, Strain Adenoid 6, the result was confirmed by the commercial kit FTD Neonatal Meningitis.

\$ It is a synthetic sample in faecal matrix and may carry DNA from *E. coli*. For Norovirus GI, the positive result for *E. coli* was confirmed by the commercial kit FTD Neonatal Meningitis.

Shigella and *E. coli* share 80-90% of sequence homology and are difficult to distinguish by PCR.

& It is a synthetic sample in faecal matrix and may carry DNA from *Enterobacter*.

12.4. Analytical reactivity

The reactivity of VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit for *Enterobacter* was confirmed by Real Time PCR using DNA extracted from *Enterobacter cloacae* subs. *cloacae* strain CDC 442-68, (as template), showing positive results.

The reactivity of VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit for *A. baumannii* was confirmed by Real Time PCR using DNA extracted from *Acinetobacter baumannii* strain 5377 (as template), showing positive results.

The reactivity of VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit for *E. coli* was confirmed by Real Time PCR using DNA extracted from *Escherichia coli* strains 1101362 (which expresses KPC) and 1001728 (which expresses NDM-1); from strains of *Escherichia coli* expressing IMP, MCR-1, and CTX-M-15, OXA-1 and TEM-1; and from strains of enteroinvasive *Escherichia coli* (which expresses *ipaH* gene), enteropathogenic *Escherichia coli* (which expresses *eae* gene), enterohemorrhagic *Escherichia coli* serotype O157:H7 and enterotoxigenic *Escherichia coli* serotype O25:H42 (as templates), showing positive results.

ANNEX 1

OPEN AND ROTOR-GENE FORMAT WITH INTERNAL CONTROL

Annex for the following references:

PRODUCT	REFERENCES
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 6 x 8-well strips, low profile	VS-EAC106L
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 6 x 8-well strips, high profile	VS-EAC106H
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 12 x 8-well strips, low profile	VS-EAC112L
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 12 x 8-well strips, high profile	VS-EAC112H
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 96-well plate, low profile	VS-EAC113L
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 96-well plate, high profile	VS-EAC113H
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 9 x 4-well strips, Rotor-Gene®	VS-EAC136
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 18 x 4-well strips, Rotor-Gene®	VS-EAC172
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 1 x 8-well strips, low profile	VS-EAC101L
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 1 x 8-well strips, high profile	VS-EAC101H
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 2 x 4-well strips, Rotor-Gene®	VS-EAC101

Table A1 1. References

A1.1 Principle of the procedure

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer and polymerase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity.

Target	Channel	Gene
<i>Enterobacter</i>	FAM	Xx and <i>Omp</i> genes
<i>A. baumannii</i>	ROX	<i>bap</i> gene
<i>E. coli</i>	Cy5	<i>16S rRNA</i> gene
Internal Control (IC)	HEX, VIC or JOE *	-

Table A1 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult the website www.cerfest.es.

A1.2 Reagents provided

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit includes the following materials and reagents detailed in Tables A1.3, A1.4 and A1.5. Based on the commercial presentation and the Real Time PCR platform used, the stabilized PCR reaction mix could be placed inside different wells and could be marketed on multiple formats. Table A1.3 includes materials and reagents to be used with 8-well strips compatible devices. Table A1.4 includes materials and reagents to be used with 96-well plate compatible devices. Table A1.5 includes

materials and reagents for use with Qiagen/Corbett Rotor-Gene® instruments for 4-well strips. (Consult the thermocycler compatibility on CerTest's website www.cer-test.es).

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and internal control in stabilized format	White	1/6/12 x 8-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	1/6/12 x 8-cap strip

Table A1 3. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii* & *E. coli* Real Time PCR Detection Kit with Ref. VS-EAC101L, VS-EAC101H, VS-EAC106L, VS-EAC106H, VS-EAC112L and VS-EAC112H.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and internal control in stabilized format	White	1 plate
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 x 8-cap strip

Table A1 4. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii* & *E. coli* Real Time PCR Detection Kit with Ref. VS-EAC113L and VS-EAC113H.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> 4-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and internal control in stabilized format	Transparent	2/9/18 x 4-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii</i> & <i>E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
4-cap strips	Caps for sealing wells during thermal cycling	Transparent	2/9/18 x 4-cap strip

Table A1 5. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii* & *E. coli* Real Time PCR Detection Kit with Ref. VS-EAC101, VS-EAC136 and VS-EAC172. For use with Qiagen/Corbett Rotor-Gene® instruments and compatible accessories with strips of 4 tubes 0.1 ml (72-Well Rotor and Locking Ring 72-Well Rotor).

A1.3 Test procedure

A1.3.1 Lyophilized positive control

Enterobacter, *A. baumannii* & *E. coli* Positive Control contains high copies of the template; the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A1.3.2 PCR protocol

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run for each assay. Peel off protective aluminium seal from strips.

- 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

- 2) Adding samples and controls.

Add 5 µL of DNA sample, reconstituted *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) or Negative Control (violet vial) in different wells and close them with the provided caps. It is recommended to briefly centrifuge the 8-well strips or gently tap each strip onto a hard surface to ensure that all the liquids are at the bottom of the tubes (for Qiagen/Corbett Rotor-Gene®).

Load the strips or plate in the thermocycler.

- 3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A1 6. PCR protocol

Fluorogenic data should be collected during the extension step (*) through the FAM (*Enterobacter*), ROX (*A. baumannii*), Cy5 (*E. coli*) and HEX, JOE or VIC (Internal Control (IC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 2

TUBE FORMAT WITH INTERNAL CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit, 4 tubes x 24 reactions	VS-EAC196T

Table A2. 1.References.

A2.1 Principle of the procedure

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains in each Reaction-Mix tube all the components necessary for 24 real time PCR reactions (specific primers/probes, dNTPs, buffer and polymerase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity.

Target	Channel	Gene
<i>Enterobacter</i>	FAM	Xx and Omp genes
<i>A. baumannii</i>	ROX	bap gene
<i>E. coli</i>	Cy5	16S rRNA gene
Internal Control (IC)	HEX, VIC or JOE *	-

Table A2. 2.Target, channel and genes.

*Depending on the equipment used select the proper detection channel, channel, to check most common detection channels consult website www.certest.es.

A2.2 Reagents provided

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table A2.3.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Reaction-Mix tube	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and internal control in stabilized format	White	4 vials
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL

Table A2. 3. Reagents and materials provided in VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit with Ref. VS-EAC196T.

A2.3 Test procedure

A2.3.1 Lyophilized positive control

Enterobacter, *A. baumannii* & *E. coli* Positive Control contains high copies of the template; the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the

lyophilized *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A2.3.2 Lyophilized reaction mix tube

Determine the number of required reactions including samples and controls (one positive and negative control must be included in each run). Obtain the correct number of lyophilized Reaction-Mix vials (24-reactions each one) for testing.

Recommendation is to open and manipulate the *Enterobacter*, *A. baumannii* & *E. coli* Reaction-Mix tube in pre-PCR laboratory area. Open lyophilized Reaction-mix tube (white vial) carefully to avoid disruption of the pellet and add 390 µL of Rehydration Buffer (blue vial) supplied. Mix gently by pipetting up and down. Spin down briefly to remove bubbles generated during mixing.

Once the Reaction-Mix tube has been re-suspended, return unused reagents to the appropriate storage conditions at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The volume of the rehydrated Reaction-Mix is sufficient for 24 reactions. The rehydrated Reaction-Mix may be kept at 25°C±5°C or 2-8°C for up to 4-hours (see Transport and storage conditions section for additional storage options).

A2.3.3 PCR protocol

- 1) Adding rehydrated Reaction-Mix to the number of required wells.

Add 15 µL of rehydrated *Enterobacter*, *A. baumannii* & *E. coli* Reaction-Mix (white vial) into each well.

- 2) Adding samples and controls.

Add 5 µL of DNA sample, reconstituted *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the tubes with caps or seal the plate. Centrifuge briefly.

Load the plate, the strips, or tubes in the thermocycler.

- 3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A2. 4. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Enterobacter*), ROX (*A. baumannii*), Cy5 (*E. coli*) and HEX, JOE or VIC (Internal Control (IC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 3

OPEN AND ROTOR-GENE FORMAT WITH EXTRACTION CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 6 x 8-well strips, low profile	VS-EAC106LE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 6 x 8-well strips, high profile	VS-EAC106HE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 12 x 8-well strips, low profile	VS-EAC112LE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 12 x 8-well strips, high profile	VS-EAC112HE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 96-well plate, low profile	VS-EAC113LE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 96-well plate, high profile	VS-EAC113HE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 9 x 4-well strips, Rotor-Gene®	VS-EAC136E
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 18 x 4-well strips, Rotor-Gene®	VS-EAC172E
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 1 x 8-well strips, low profile	VS-EAC101LE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 1 x 8-well strips, high profile	VS-EAC101HE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit 2 x 4-well strips, Rotor-Gene®	VS-EAC101E

Table A3. 1. References.

A3.1 Principle of the procedure

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPs, buffer and polymerase) in a stabilized format. The assay can use an Extraction Control (EC) which can be introduced into each sample at the lysis buffer stage of the extraction process. This control can be used to monitor the extraction process and/or discard the inhibition of the polymerase activity.

Target	Channel	Gene
<i>Enterobacter</i>	FAM	Xx and <i>Omp</i> genes
<i>A. baumannii</i>	ROX	<i>bap</i> gene
<i>E. coli</i>	Cy5	16S rRNA gene
Extraction Control (EC)	HEX, VIC or JOE *	-

Table A3. 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult website www.certest.es.

A3.2 Reagents provided

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit includes the following materials and reagents detailed in Tables A3.3, A3.4 and A3.5. Based on the commercial presentation and the Real Time PCR platform used, the stabilized PCR reaction mix could be placed inside different wells and could be marketed on multiple formats. Table A3.3 includes materials and reagents to be used with 8-well strips compatible devices.

Table A3.4 includes materials and reagents to be used with 96-well plate compatible devices. Table A3.5 includes materials and reagents for use with Qiagen/Corbett Rotor-Gene® instruments for 4-well strips. (Consult the thermocycler compatibility on CerTest's website www.cerTEST.es).

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii & E. coli</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	1/6/12 x 8-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii & E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	1/6/12 x 8-cap strip

Table A3. 3. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii & E. coli* Real Time PCR Detection Kit with Ref. VS-EAC101LE, VS-EAC101HE, VS-EAC106LE, VS-EAC106HE, VS-EAC112LE and VS-EAC112HE.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii & E. coli</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	1 plate
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii & E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 x 8-cap strip

Table A3. 4. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii & E. coli* Real Time PCR Detection Kit with Ref. VS-EAC113LE and VS-EAC113HE.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter, A. baumannii & E. coli</i> 4-well strips	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	Transparent	2/9/18 x 4-well strip
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter, A. baumannii & E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL
4-cap strips	Caps for sealing wells during thermal cycling	Transparent	2/9/18 x 4-cap strip

Table A3. 5. Reagents and materials provided in VIASURE *Enterobacter, A. baumannii & E. coli* Real Time PCR Detection Kit with Ref. VS-EAC101E, VS-EAC136E and VS-EAC172E. For use with Qiagen/Corbett Rotor-Gene® instruments and compatible accessories with strips of 4 tubes 0.1 ml (72-Well Rotor and Locking Ring 72-Well Rotor).

A3.3 Test procedure

A3.3.1 Lyophilized extraction control

Recommendation is to open and manipulate the Extraction Control (EC) in pre-PCR laboratory area away from the lyophilized positive control. Reconstitute the lyophilized Extraction Control (green vial) by adding 500 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the Extraction Control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The Water RNase/DNase free vial must be utilized first to reconstitute the lyophilized Extraction Control in pre-PCR laboratory area, and subsequently, it can be used to reconstitute the lyophilized *Enterobacter, A. baumannii & E. coli* Positive Control in an area away from the other components.

A3.3.2 Lyophilized positive control

Enterobacter, A. baumannii & E. coli Positive Control contains high copies of the template; the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Enterobacter, A. baumannii & E. coli* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A3.3.3 PCR protocol

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run for each assay. Peel off protective aluminium seal from strips.

- 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

2) Adding samples and controls.

Add 5 µL of Negative Control (violet vial) in the reserved wells for negative control.

Add 5 µL of DNA sample in different wells.

Add 5 µL of reconstituted *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) in the reserved wells for positive control.

If the EC is only used as PCR inhibition control, add 1 µL of the EC (green vial) to the negative control/sample/positive control wells.

It is recommended to briefly centrifuge the 8-well strips, or gently tap each strip onto a hard surface to ensure that all the liquids are at the bottom of the tubes (for Qiagen/Corbett Rotor-Gene® kit).

Load the strips or plate in the thermocycler.

3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A3. 6. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Enterobacter*), ROX (*A. baumannii*), Cy5 (*E. coli*) and HEX, JOE or VIC (Extraction Control (EC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

ANNEX 4

TUBE FORMAT WITH EXTRACTION CONTROL

Annex for the following references:

PRODUCT	REFERENCE
VIASURE <i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Real Time PCR Detection Kit, 4 tubes x 24 reactions	VS-EAC196TE

Table A4. 1. References.

A4.1 Principle of the procedure

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit contains in each Reaction-Mix tube all the components necessary for 24 real time PCR reactions (specific primers/probes, dNTPs, buffer and polymerase) in a stabilized format. The assay can use an Extraction Control (EC) which can be introduced into each sample at the lysis buffer stage of the extraction process. This control can be used to monitor the extraction process and/or discard the inhibition of the polymerase activity.

Target	Channel	Gene
<i>Enterobacter</i>	FAM	<i>Xx</i> and <i>Omp</i> genes
<i>A. baumannii</i>	ROX	<i>bap</i> gene
<i>E. coli</i>	Cy5	<i>16S rRNA</i> gene
Extraction Control (EC)	HEX, VIC or JOE *	-

Table A4. 2. Target, channel and genes.

*Depending on the equipment used select the proper detection channel, to check most common detection channels consult website www.certest.es.

A4.2 Reagents provided

VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table A4.3.

Reagent/Material	Description	Colour	Amount
<i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Reaction-Mix tube	A mix of enzymes, primers probes, buffer, dNTPs and stabilizers in stabilized format	White	4 vials
Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
<i>Enterobacter</i> , <i>A. baumannii</i> & <i>E. coli</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
Extraction Control	Non-infectious nucleic acid lyophilized	Green	1 vial
Negative control	Non template control	Violet	1 vial x 1 mL
Water RNase/DNase free	RNase/DNase free water	White	1 vial x 1 mL

Table A4. 3. Reagents and materials provided in VIASURE *Enterobacter*, *A. baumannii* & *E. coli* Real Time PCR Detection Kit with Ref. VS-EAC196TE.

A4.3 Test procedure

A4.3.1 Lyophilized extraction control

Recommendation is to open and manipulate the Extraction Control (EC) in pre-PCR laboratory area away from the lyophilized positive control. Reconstitute the lyophilized Extraction Control (green vial) adding 500 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the Extraction Control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The Water RNase/DNase free vial must be utilized first to reconstitute the lyophilized Extraction Control in pre-PCR laboratory area, and subsequently, it can be used to reconstitute the lyophilized *Enterobacter*, *A. baumannii* & *E. coli* Positive Control in an area away from the other components.

A4.3.2 Lyophilized positive control

Enterobacter, *A. baumannii* & *E. coli* Positive Control contains high copies of the template; the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) by adding 100 µL of the supplied Water RNase/DNase free (white vial) and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. It is recommended to separate it in aliquots to minimize freeze and thaw cycles.

A4.3.3 Lyophilized reaction mix tube

Determine the number of required reactions including samples and controls (one positive and negative control must be included in each run). Obtain the correct number of lyophilized Reaction-Mix vials (24-reactions each one) for testing.

Recommendation is to open and manipulate the *Enterobacter*, *A. baumannii* & *E. coli* Reaction-Mix tube in pre-PCR laboratory area. Open lyophilized Reaction-mix tube (white vial) carefully to avoid disruption of the pellet and add 390 µL of Rehydration Buffer (blue vial) supplied. Mix gently by pipetting up and down. Spin down briefly to remove bubbles generated during mixing.

Once the Reaction-Mix tube has been re-suspended, return unused reagents to the appropriate storage conditions at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

Note: The volume of the rehydrated Reaction-Mix is sufficient for 24 reactions. The rehydrated Reaction-Mix may be kept at 25°C±5°C or 2-8°C for up to 4-hours (see Transport and storage conditions section for additional storage options).

A4.3.4 PCR protocol

- 1) Adding rehydrated Reaction-Mix to the number of required wells.

Add 15 µL of rehydrated *Enterobacter*, *A. baumannii* & *E. coli* Reaction-Mix (white vial) into each well.

- 2) Adding samples and controls.

Add 5 µL of Negative Control (violet vial) in the reserved wells for negative control.

Add 5 µL of DNA sample in different wells.

Add 5 µL of reconstituted *Enterobacter*, *A. baumannii* & *E. coli* Positive Control (red vial) in the reserved wells for positive control.

If the EC is only used as PCR inhibition control, add 1 µL of the EC (green vial) to the negative control/sample/positive control wells.

Close the wells with the caps provided. Centrifuge briefly.

Load the plate, the strips or tubes in the thermocycler.

- 3) Set up the thermocycler (consult thermocycler compatibility on CerTest's website www.certest.es).

Program the thermocycler following the conditions listed below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturation	10 sec	95°C
	Annealing/Extension (Data collection*)	50 sec	60°C

Table A4. 4. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Enterobacter*), ROX (*A. baumannii*), Cy5 (*E. coli*) and HEX, JOE or VIC (Extraction Control (EC)). Depending on the equipment used select the proper detection channel (to check most common detection channels consult website www.certest.es).

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Bibliography/Bibliografía

1. Antimicrobial resistance | World Health Organization. <https://www.who.int/news-room/fact-sheets/detail/antimicrobial-resistance>.
2. Brenner, D. J., Fanning, G. R., Steigerwalt, A. G., Orskov, I., & Orskov, F. (1972). Polynucleotide sequence relatedness among three groups of pathogenic *Escherichia coli* strains. *Infection and Immunity*, 6(3), 308–315. <https://doi.org/10.1128/IAI.6.3.308-315.1972>
3. Davin-Regli A, Lavigne J-P, Pagès J-M. *Enterobacter* spp.: update on taxonomy, clinical aspects, and emerging antimicrobial resistance. *Clin Microbiol Rev*. 2019;32(4):e00002-19. doi:10.1128/CMR.00002-19.
4. Devanga Ragupathi, N. K., Muthuirulandi Sethuvel, D. P., Inbanathan, F. Y., & Veeraraghavan, B. (2018). Accurate differentiation of *Escherichia coli* and *Shigella* serogroups: challenges and strategies. *New Microbes and New Infections*, 21, 58–62. <https://doi.org/10.1016/j.nmni.2017.09.003>
5. Kyriakidis I, Vasileiou E, Pana ZD, Tragiannidis A. *Acinetobacter baumannii* antibiotic resistance mechanisms. *Pathogens*. 2021;10(3):1-31. doi:10.3390/pathogens10030373.
6. Ud-Din, A., & Wahid, S. (2014). Relationship among *Shigella* spp. and enteroinvasive *Escherichia coli* (EIEC) and their differentiation. *Brazilian Journal of Microbiology*, 45(4), 1131–1138. <https://doi.org/10.1590/S1517-83822014000400002>
7. Vázquez-López R, Solano-Gálvez SG, Vignon-Whaley JJJ, et al. *Acinetobacter baumannii* resistance: A real challenge for clinicians. *Antibiotics*. 2020;9(4):1-22. doi:10.3390/antibiotics9040205.
8. Da Silva GJ, Mendonça N. Association between antimicrobial resistance and virulence in *Escherichia coli*. *Virulence*. 2012;3(1):18-28. doi:10.4161/viru.3.1.18382.
9. WHO | World Health Organization. *E. coli*. Published 2018. <https://www.who.int/news-room/fact-sheets/detail/e-coli>.
10. Van Den Beld, M. J. C., & Reubsaet, F. A. G. (2012). Differentiation between *Shigella*, enteroinvasive *Escherichia coli* (EIEC) and noninvasive *Escherichia coli*. *European Journal of Clinical Microbiology and Infectious Diseases*, 31(6), 899–904. <https://doi.org/10.1007/s10096-011-1395-7>
11. De Oliveira DMP, Forde BM, Kidd TJ, et al. Antimicrobial resistance in ESKAPE pathogens. *Clin Microbiol Rev*. 2020;33(3):1-49. doi:10.1128/CMR.00181-19.
12. Poirer L, Madec J-Y, Lupo A, et al. Antimicrobial Resistance in *Escherichia coli*. *Microbiol Spectr*. 2018;6(4). doi:10.1128/MICROBIOLSPEC.ARBA-0026-2017

Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico *in vitro*

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Trademarks

Modification rights reserved. All rights reserved. © CerTest Biotec, S.L.

All other trademarks that may appear in this package insert are the property of their respective owners.

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