CHORUS

TOXOPLASMA IgA

REF 81043

REF 81043/12

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16. BIBLIOGRAFIA

1. J. Husskinson, P. Stepick-Biek et al.: Toxoplasma antigens recognized by immunoglobulin G subclass during acute and chronic infection. J. Clin. Microbiol. 27: 2031 (1989)
2. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
3. A.M. Van Loon and J. Van der Veen: Enzyme-linked immunosorbent assay for quantitation of toxoplasma antibodies in human sera. J. Clin. Pathol. 33: 635 (1980).
4. P. Stepick-Biek et al.: Contribution of specific serum IgA assay to the diagnosis of Toxoplasmosis. Ann. Pediatr. 34: 375 (1987).
5. E. Rigoli et al.: Determinazione e significato delle IgA anti-Toxoplasma gondii nell'identificazione della fase esordiente dell'infezione toxoplasmica. Microbiologia Medica 5: 95 (1990).

INSTRUCTIONS FOR USE

CHORUS TOXOPLASMA IgA

(English)

1. INTENDED USE

QUALITATIVE DETERMINATION OF IgA-CLASS ANTIBODIES ANTI TOXOPLASMA GONDII IN HUMAN SERUM, USING A DISPOSABLE DEVICE APPLIED ON THE CHORUS INSTRUMENTS

2. INTRODUCTION

Toxoplasma gondii is a ubiquitous protozoa which causes infection in all species of mammals. In man, serological studies have shown that a high percentage of the adult population becomes infected by this parasite. The disease is particularly dangerous during pregnancy, being transmitted from mother to fetus and causing serious malformations. For this reason it is important to determine the antibody titer at the beginning of pregnancy; those subjects who do not present immunity should be followed up with frequent analyses to check for serum conversion. The assay of the IgA is of particular importance in establishing serologically the acute phase of the disease.

3. PRINCIPLE OF THE METHOD

The test is based on the ELISA principle (Enzyme linked Immunosorbent Assay).

The antigen is bound to the solid phase. The specific immunoglobulins are bound to the antigen through incubation with dilute human serum.

After washings to eliminate the proteins which have not reacted, incubation is performed with the conjugate, composed of human IgA monoclonal antibodies conjugated to peroxidase.

The unbound conjugate is eliminated, and the peroxidase substrate added.

The density of the colour which develops is proportional to the concentration of specific antibodies present in the serum sample.

The disposable devices contain all the reagents to perform the test when applied on the Chorus instruments.

4. KIT COMPOSITION AND REAGENT PREPARATION

The kit is sufficient for 36 tests (REF 81043).

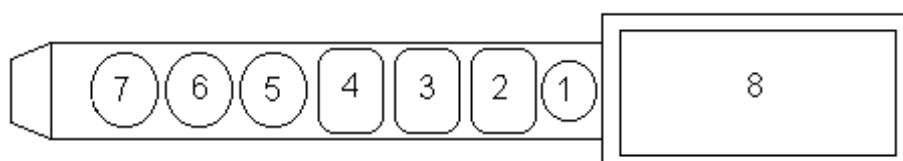
The kit is sufficient for 12 tests (REF 81043/12).

DD DEVICES

6 packages each containing 6 devices (REF 81043).

2 packages each containing 6 devices (REF 81043/12).

Use: equilibrate a package at room temperature, open the package and remove the devices required; replace the others in the bag with the silica gel, expel the air and seal by pressing the closure. Store at 2-8°C.

Description:

Position 8: Space for application of bar code label

Position 7: empty

Position 6: MICROPLATE WELL coated with *Toxoplasma Gondii*

Position 5: Uncoated MICROPLATE WELL

Position 4: TMB SUBSTRATE

Contents: Tetramethylbenzidine 0.26 mg/mL and H₂O₂ 0.01% stabilized in 0.05 mol/L citrate buffer (pH 3.8)

Position 3: SAMPLE DILUENT Contents: Proteic solution containing phenol 0.05%, Bronidox 0.02% and an indicator to reveal the presence of the serum.

Position 2: CONJUGATE

Contents: anti-human IgA monoclonal antibodies labelled with horseradish peroxidase, in phosphate buffer containing phenol 0.05% and Bronidox 0.02%

Position 1: EMPTY WELL in which the operator must place the undiluted serum

CALIBRATOR CALIBRATOR 1 x 0.175 mL

Contents: Diluted human serum, at known antibody concentration, containing Proclin and Gentamycin, **liquid ready for use.**

CONTROL + POSITIVE CONTROL 1 x 0.425 mL

Contents: Diluted human serum, at known antibody concentration, containing Proclin and Gentamycin, **liquid ready for use.**

MATERIALS REQUIRED BUT NOT PROVIDED.

- WASHING BUFFER **REF** 83606
- CLEANING SOLUTION 2000 **REF** 83609
- SANITIZING SOLUTION **REF** 83604 – 83608
- CHORUS NEGATIVE CONTROL/SAMPLE DILUENT **REF** 83607
- Chorus / Chorus Trio Instrument
- Distilled or deionised water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 50-200 µl solution
- Disposable gloves
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials

5. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C. In the case of storage at an incorrect temperature the calibration must be repeated and the run validated using the control serum (see point 9, Test validation).

The expiry date is printed on each component and on the kit label.

Reagents have a limited stability after opening and/or preparation:

DEVICES	8 weeks at 2/8°C
POSITIVE CONTROL	8 weeks at 2/8°C
CALIBRATOR	8 weeks at 2/8°C

6. PRECAUTIONS

ONLY FOR DIAGNOSTIC USE IN VITRO.

Attention:

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HbsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling all the reagents and samples.

Waste disposal: serum samples, calibrators and strips once used must be treated as infectious residuals and eliminated according to law.

Health and Safety Information

1. Do not pipette by mouth. Wear disposable gloves and eye protection while handling specimens. Wash hands thoroughly after placing the devices in the CHORUS instrument.
2. The following reagents contain low concentrations of harmful or irritant substances:
 - a) The conjugate contains phenol
 - b) The substrate is acid
 If any of the reagents come into contact with the skin or eyes, wash the area extensively with water.
3. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1.0%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.
4. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1.0% sodium hypochlorite before work is continued. Sodium

hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical precautions

Bring the devices to room temperature (18-30°C) before use; use within 60 min.

1. **Discard devices which show the substrate (well 4) blue colored.**
2. Adding the sample into the well verify that it is perfectly distributed on the bottom.
3. Check for the presence of the reagents in the device and that the device is not damaged; do not use devices which are lacking a reagent.
4. The devices are for use with the Chorus instrument; the instructions for use must be carefully followed and the instrument operating manual must be consulted.
5. Check that the Chorus instrument is set up correctly (see Chorus Operating Manual).
6. Do not alter the bar code placed on the handle of the device in order to allow correct reading by the instrument
7. Defective barcodes can be inserted manually in the instrument.
8. Do not expose the devices to strong light or to hypochlorite vapours during storage and use.
9. The use of strongly hemolyzed samples, or samples presenting microbial contamination may all constitute sources of error.
10. Before inserting the devices in the instrument, check that the reaction well does not contain foreign bodies.
11. Pipette the test serum (50 µl) in well 1 of the device (see figure).
12. Do not use the device after the expiry date

7. TYPE AND STORAGE OF SAMPLE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice. The fresh serum may be stored for 4 days at 2/8°C, or frozen for longer periods at -20°C, and can be thawed a maximum of 3 times. Do not keep the samples in auto-defrosting freezers. Defrosted samples must be shaken carefully before use. The quality of the sample can be seriously affected by microbial contamination which leads to erroneous results.

Strongly lipemic, icteric, hemolyzed or contaminated samples should be avoided.

The test cannot be applied to plasma.

8. PROCEDURE

1. Open the package (on the side containing the pressure-closure), remove the number of devices required and seal the rest in the bag after expelling the air.
2. Check the state of the device according to the indications reported in the Analytical Warnings points 1 and 8.
3. Dispense 50 µl of undiluted test serum in well no. 1 of each device; at each change of batch, use a device for the calibrator.
4. Place the devices in the Chorus instrument. Perform the calibration (if necessary) and the test as reported in the Chorus Operating Manual.

9. TEST VALIDATION

Use the control serum to check the validity of the results obtained. It should be used as reported in the operating manual. If the instrument signals that the control serum has a value outside the acceptable range, the calibration must be repeated. The previous results will be automatically corrected.

If the result of the control serum continues to be outside the acceptable range, contact the Scientific Support.

Tel: 0039 0577 319554
 Fax: 0039 0577 366605
 email: scientificsupport@diesse.it

10. INTERPRETATION OF THE RESULTS

The Chorus instrument gives the result as an INDEX (ratio between the O.D. value of the examined samples and the O.D. value of the Cut-off) which can be used as the quantitative measurement since it is proportional to the quantity of specific IgA.

The sample will give the following results:

NEGATIVE: when the result is < 0.8

DOUBTFUL/EQUIVOCAL: when the result is between 0.8 and 1.2.

POSITIVE: when the ratio is > 1.2

In the case of a doubtful/equivocal result, repeat the test. If it remains doubtful/equivocal, take a new sample.

11. LIMITATIONS OF THE PROCEDURE

A second serum sample obtained 8-14 days later, should be tested in parallel to determine an increase in the IgA level. For a complete diagnosis the IgM level should be determined using the IgM kit (Ref. 81041). The test result should be used in conjunction with information available from other diagnostic procedures.

12. ANALYTICAL SPECIFICITY

3 samples (1 Negative, 1 Cut-Off and 1 Positive) were spiked with the following potentially interfering factors and then tested:

Rheumatoid Factor (44 – 220 IU/ml)
 Bilirubin (4.5 mg/dl – 45 mg/dl)
 Triglycerides (10 mg/dl – 250 mg/dl)
 Hemoglobin (5 mg/ml – 30 mg/ml)

The presence in the serum sample of the interfering substances described above does not affect the test result.

13. CROSS-REACTIONS

11 samples, positive to Mumps virus, Epstein-Barr virus, Rubella virus, Cytomegalovirus, Herpes Simplex virus (type1) and ANA were tested.

No significant cross-reactions were found.

14. DIAGNOSTIC SENSITIVITY AND SPECIFICITY

54 samples were tested in an experimentation: the samples were tested with another commercial immunoenzymatic method.

The results are summarized in the following table:

Table 1.

		REFERENCE	
		+	-
DIESSE	+	28	0
	-	0	26

The kit shows a sensitivity of 100% (CI 95%: 88–100) and a specificity of 100% (CI 95%: 87–100).

15. PRECISION

Table 2. Within-run precision

	No. of replicates	Mean Index	St. Dev.	CV%
Lot no. 012	8	2.5	0.11	4
Lot no. 013	6	3.1	0.19	6
Lot no. 014	9	2.9	0.17	6

Table 3. Between-run precision and precision between batches

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16. REFERENCES

1. J. Husskinson, P. Stepick-Biek et al.: Toxoplasma antigens recognized by immunoglobulin G subclass during acute and chronic infection. J. Clin. Microbiol. 27: 2031 (1989).
2. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
3. A.M. Van Loon and J. Van der Veen: Enzyme-linked immunosorbent assay for quantitation of toxoplasma antibodies in human sera. J. Clin. Pathol. 33: 635 (1980).
4. P. Stepick-Biek et al.: Contribution of specific serum IgA assay to the diagnosis of Toxoplasmosis. Ann. Pediatr. 34: 375 (1987).
5. E. Rigoli et al.: Determinazione e significato delle IgA anti-Toxoplasma gondii nell'identificazione della fase esordiente dell'infezione toxoplasmica. Microbiologia Medica 5: 95 (1990).

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17. POUŽITÁ LITERATURA

5. J. Husskinson, P. Stepick-Biek et al.: Toxoplasma antigens recognized by immunoglobulin G subclass during acute and chronic infection. J. Clin. Microbiol. 27: 2031 (1989).
6. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
7. A.M. Van Loon and J. Van der Veen: Enzyme-linked immunosorbent assay for quantitation of toxoplasma antibodies in human sera. J. Clin. Pathol. 33: 635 (1980).
8. P. Stepick-Biek et al.: Contribution of specific serum IgA assay to the diagnosis of Toxoplasmosis. Ann. Pediatr. 34: 375 (1987).
5. E. Rigoli et al.: Determinazione e significato delle IgA anti-Toxoplasma gondii nell'identificazione della fase esordiente dell'infezione toxoplasmica. Microbiologia Medica 5: 95 (1990).

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17. REFERINTE

- 1.J. Husskinson, P. Stepick-Biek et al.: Toxoplasma antigens recognized by immunoglobulin G subclass during acute and chronic infection. J. Clin. Microbiol. 27: 2031 (1989).
- 2.G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
- 3.A.M. Van Loon and J. Van der Veen: Enzyme-linked immunosorbent assay for quantitation of toxoplasma antibodies in human sera. J. Clin. Pathol. 33: 635 (1980).
- 4.P. Stepick-Biek et al.: Contribution of specific serum IgA assay to the diagnosis of Toxoplasmosis. Ann. Pediatr. 34: 375 (1987).
- 5.E. Rigoli et al.: Determinazione e significato delle IgA anti-Toxoplasma gondii nell'identificazione della fase esordiente dell'infezione toxoplasmica. Microbiologia Medica 5: 95 (1990).

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