



**DIRUI<sup>®</sup>**

(Concentrated type)

## Instruction

**Product Name**

Cleaning liquid

**Volume**

200ml

**Intended Use**

It is used for cleaning and rinsing of tubing system. It will effectively clean the remained urine and other test-interfering particles to avoid cross-contamination among urine sample.

**Main Ingredient**

Lowfoaming surfactant

**For Instrument**

Automatic Urine Analyzer

**Usage**

1. Pour the detergent into a clean barrel with volume of 10L.
2. Rinse the detergent bottle with distilled water, and pour the water into the 10L barrel.
3. Dilute the detergent to 10L level with distilled water and mix it.
4. Connect the 10L barrel with Automatic Urine Analyzer, please refer to the Automatic Urine Analyzer user manual for connection method.

**Important Notice**

1. This detergent is used for Automatic Urine Analyzer only, not applicable to other instruments.
2. It should be used together with Automatic Urine Analyzer under room temperature.
3. The detergent contains chemical medicine, like preservative, etc. which can not be swallowed. Rinse with water immediately if the detergent contacts your skin.

**Storage Condition and Shelf Life**

Stored at 2°C~30°C, sealed, the shelf life is 1 year. its validity after being diluted is 1 month at 2°C~30°C.

**Manual Approved and Modified Date**

12/2018

**Symbol Explanation**

Please read instruction



Storage Temperature



Expiry



Catalogue number



Manufacturer



Authorised Representative



In Vitor Diagnosis



European In Vitro Diagnostic Medical Device Directive 98/79/EC(IVDD)



Lot No



**DIRUI INDUSTRIAL CO.,LTD.**

95 Yunhe Street, New & High Tech.

Development Zone Changchun, Jilin 130012 P.R.China

Tel: +86 431 85100409

Fax: +86 431 85172581

E-mail: dirui@dirui.com.cn

http://www.dirui.com.cn



EMERGO EUROPE Prinsessegracht 20  
2514 AP The Hague The Netherlands