CHORUS

a-TG

REF 86072

REF 86072/12

DIESSE Diagnostica Senese
S.p.A.
Via delle Rose, 10
53035 Monteriggioni (Siena)
Italy

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INSTRUCTIONS FOR USE

CHORUS a-TG

For the quantitative determination of IgG antibodies anti-Thyroglobulin (Tg)

For *In Vitro* Diagnostic Use Only

1. INTENDED USE

Immunoenzymatic method for the quantitative determination of IgG class antibodies against Thyroglobulin (Tg) in human serum, using a disposable device applied on the chorus and Chorus TRIO instruments.

2. INTRODUCTION

Thyroglobulin (Tg) is a glycoprotein of high molecular weight (660kDa) localized within the colloid of the thyroid follicle. It plays an essential role in the storage of iodine and acts as substratum for the synthesis of iodinated thyroid hormones thyroxine (T4) and 3,5,3 – triiodothyronine (T3).

Tg is one out of three major thyroid autoantigens (besides thyroid peroxidase (TPO) and the TSH-receptor). Detecting autoantibodies to Tg today is an established tool for diagnosing chronic autoimmune thyroiditis as well as for the differential diagnosis of hypothyroidism including its subclinical or latent type.

Autoantibodies to Tg and TPO are important for ruling out autoimmune thyroid diseases, as over 98% of thyroiditis patients display autoantibodies to either one or both of these antigens. Anti-Tg is found in 30% of patients with Graves disease and 85% of patients with Hashimoto's thyroiditis. Thus negative testing for anti-Tg and anti-TPO can virtually exclude a diagnosis of thyroiditis. These autoantibodies do also occur in patients with adenocarcinoma of the thyroid or hyperthyroidism. Approximately 10% of healthy individuals have anti-Tg antibodies at low levels. A follow-up to a 20-year community study in England concluded that a primary risk factor to future autoimmune thyroid dysfunction was a positive thyroid autoantibody test. Therefore these antibodies have a predictive value, too.

3. PRINCIPLE OF THE METHOD

The Chorus a-Tg device is ready to use for the detection of IgG antibodies against Tg, in the Chorus/Chorus TRIO instruments. The test is based on the ELISA principle (Enzyme linked Immunosorbent Assay). The antigen is bound to the solid phase. The specific immunoglobulins are bound to the antigen

through incubation with diluted human serum. After washing to eliminate the proteins which have not reacted, incubation is performed with the conjugate, composed of anti-human immunoglobulins antibodies conjugated to horse radish peroxidase.

The unbound conjugate is eliminated and the peroxidase substrate is added. The colour which develops is proportional to the concentration of specific antibodies present in the serum sample.

The disposable devices contain all the reagents to perform the test in the Chorus/Chorus TRIO instruments.

The results are expressed in International Units (IU/ml) calculated in reference to WHO 65/093.

4. WARNINGS AND PRECAUTIONS

FOR *IN VITRO* DIAGNOSTIC USE ONLY

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HBsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling reagents and samples.

Waste disposal: serum samples, calibrators and strips, once used, must be treated as infectious residuals and eliminated according to law.

Health and Safety Information

1. Do not pipette by mouth.
2. Wear disposable gloves and eye protection while handling specimens.
3. Wash hands thoroughly after placing the devices in the Chorus/Chorus TRIO instrument.
4. Consult the relative Material Safety Data Sheet (available on request) for all the information on safety concerning the reagents contained in the kit.
5. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.
6. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1% sodium hypochlorite before work is continued. Sodium hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical Precautions

IO-09/257-C IFU 86072 – 86072/12 – Ed. 07.09.2015

Bring the devices to room temperature (18-30°C) before use; use within 60 min.

1. **Discard devices which show the substrate (well 4) blue colored.**
2. Adding the sample into the well verify that it is perfectly distributed on the bottom.
3. Check for the presence of the reagents in the device and that the device is not damaged. Do not use devices which are lacking a reagent and/or present foreign bodies in the reaction well when visually inspected.
4. The devices are for use with the Chorus/Chorus TRIO instrument; the Instructions for Use and the Instrument Operating Manual must be carefully followed.
5. Check that the Chorus/Chorus TRIO instrument is set up correctly (see Operating Manual).
6. Do not alter the bar code placed on the handle of the device in order to allow correct reading by the instrument.
7. Avoid using self-defrosting freezers for the storage of the samples.
8. Defective barcodes can be inserted manually in the instrument (see Operating Manual).
9. Do not expose the devices to strong light or to hypochlorite vapors during storage and use.
10. The use of strongly hemolyzed, lipemic, icteric samples, of serum not completely coagulated or of samples presenting microbial contamination may all constitute sources of error.
11. Do not use the device after the expiry date.
12. **Make sure that the instrument is connected to the Washing Buffer Autoimmunity (Ref. 86004).**

5. KIT COMPOSITION AND REAGENT PREPARATION

The kit is sufficient for 36 tests (REF 86072).

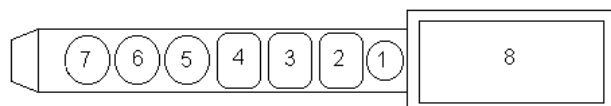
The kit is sufficient for 12 tests (REF 86072/12).

DD DEVICES

6 packages each containing 6 devices (REF 86072).

2 packages each containing 6 devices (REF 86072/12).

Description:

Position 8: Space for application of bar code label

Position 7: Empty

Position 6: MICROPLATE WELL

Coated with highly purified thyroglobulin

Position 5: Uncoated MICROPLATE WELL

Position 4: TMB SUBSTRATE

Contents: Tetramethylbenzidine 0.26 mg/mL and H₂O₂ 0.01% stabilized in 0.05 mol/L citrate buffer (pH 3.8)

Position 3: SAMPLE DILUENT

Contents: saline proteic solution with Proclin (0.1%)

Positon 2: CONJUGATE

Contents: anti-human IgG monoclonal antibodies labeled with horse radish peroxidase, in phosphate buffer containing phenol 0.05% and Bronidox 0.02%.

Position 1: EMPTY WELL

in which undiluted serum must be added

Use: equilibrate a package at room temperature, open the package and remove the required devices; replace the others in the bag with the silica gel, expel the air and **seal** by pressing the closure. Store at 2-8°C.

CALIBRATOR CALIBRATOR 1 x 0.175 ml

Contents: Diluted human serum containing IgG antibodies anti-Thyroglobulin and preservative. Liquid, ready for use.

CONTROL + POSITIVE CONTROL 1 x 0.425 ml

Contents: Diluted human serum containing IgG antibodies anti-Thyroglobulin and preservative. Liquid, ready for use.

MATERIALS REQUIRED BUT NOT PROVIDED:

- WASHING BUFFER AUTOIMMUNITY (REF 86004)
- CLEANING SOLUTION 2000 (REF 83609)
- SANITIZING SOLUTION (REF 83604 - 83608)
- CHORUS NEGATIVE CONTROL/SAMPLE DILUENT (REF 83607)
- Chorus/Chorus TRIO Instrument
- Distilled or deionised water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 50-200 µl solution
- Disposable gloves
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials

6. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C. In the case of storage at an incorrect temperature the calibration must be repeated and the run validated using the control serum (see section 9, Test validation).

The expiry date is printed on each component and on the kit label.

Reagents have a limited stability after opening:

DEVICES	8 weeks at 2/8°C
CALIBRATOR	8 weeks at 2/8°C
POSITIVE CONTROL	8 weeks at 2/8°C

7. SPECIMEN COLLECTION AND STORAGE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice.

Possible consequences, in case of use of other biological liquids, are not known.

The fresh serum may be stored for 4 days at 2/8°C, or frozen for longer periods at -20°C, and can be thawed a maximum of 3 times.

Do not keep the samples in auto-defrosting freezers. Defrosted samples must be shaken carefully before use.

Heat-inactivation can rise to erroneous results. The quality of the sample can be seriously affected by microbial contamination which leads to erroneous results.

8. ASSAY PROCEDURE

1. Open the package (on the side containing the pressure-closure), remove the number of devices required and seal the rest in the bag after expelling the air.
2. Check the state of the device according to the indications reported in chapter 4, Analytical Precautions.
3. Dispense 50 µl of undiluted test serum in well no. 1 of each device; at each change of batch, use a device for the calibrator.
4. Place the devices in the Chorus/Chorus TRIO instrument. Perform the calibration (if necessary) and the test as reported in the instrument Operating Manual.

9. TEST VALIDATION

Use the control serum to check the validity of the results obtained. It should be used as reported in the instrument Operating Manual. If the instrument signals that the control serum has a value outside the acceptable range, the calibration must be repeated. The previous results will be automatically corrected.

If the result of the control serum continues to be outside the acceptable range, contact the Scientific Support.

Tel: 0039 0577 319554
Fax: 0039 0577 366605
email: scientificsupport@diesse.it

10. INTERPRETATION OF THE RESULTS

The Chorus/Chorus TRIO instrument expresses the result in International Units (IU/ml) (International reference serum WHO 65/093), calculated on the basis of a lot-dependent graph stored in the instrument.

The test on the examined serum can be interpreted as follows:

POSITIVE: when the result is > 71.5 IU/ml

NEGATIVE: when the result is < 58.5 IU/ml

DOUBTFUL/EQUIVOCAL: for all values between 58.5 IU/ml and 71.5 IU/ml

If the result is doubtful/equivocal, repeat the test. If it remains doubtful/equivocal, collect a new serum sample.

11. LIMITATIONS

All the values obtained require a careful interpretation that must consider other indicators relative to the patient.

The test, indeed, can not be used alone for a clinical diagnosis and the test result should be evaluated together with the patient history and other clinical diagnostic evaluation.

12. CALIBRATION RANGE

Calibration range 20-1000 IU/ml.

For samples > 1000 IU/ml retest the diluted sample in the Negative Control/Sample Diluent (PF83607-not supplied with the kit).

13. REFERENCE RANGE

Among the normal population the expected values, which have been determined by examining 120 sera from healthy donors, were between 16.0 and 58.8 IU/ml.

14. ANALITICAL SPECIFICITY

5 samples (2 Negative, 1 Cut-Off and 2 Positive) were spiked with the following potentially interfering factors and then tested:

Rheumatoid factor (44-220 IU/ml)

Bilirubin (4.5 mg/dl - 45 mg/dl)

Triglycerides (10 mg/dl - 250 mg/dl)

Hemoglobin (5 mg/ml - 30 mg/ml)

The presence in the serum sample of the interfering substances described above does not affect the test result.

15. CROSS-REACTIONS

15 samples, positive to Gliadin IgG/IgA, pANCA, cANCA, dsDNA, SS-A, Sm, U1-70 RNP, a-TPO were tested.

No significant cross-reactions were found.

16. METHOD COMPARISON

In an experimentation 202 samples have been tested with Diesse kit and with a competitor kit.

Data are summarized in the following table:

		Reference		
		+	-	Tot.
Diesse	+	76	3	79
	-	5	118	123
	Tot.	81	121	202

Percent Positive Agreement (~Diagnostic Sensitivity):

93.8% CI_{95%}: 86.3-97.3

Percent Negative Agreement (~Diagnostic Specificity):

97.5% CI_{95%}: 93.0-99.1

The agreement between the two methods is excellent with a Cohen's Kappa of 0.91.

17. PRECISION AND REPEATABILITY

Sample	Within run		Between run	
	Mean IU/ml	CV%	Mean IU/ml	CV%
1	25.1	8.6	24.4	5.8
2	30.4	7.6	29.8	5.3
3	55.1	6.6	57.1	6.7
4	50.0	7.3	53.4	6.0
5	72.3	6.0	74.2	6.4
6	97.1	12.2	95.7	9.8
7	169.0	4.0	164.5	8.3
8	126.3	10.9	118.0	8.4
9	394.7	4.7	361.9	12.4

Sample	Between lots		Between Instruments	
	Mean IU/ml	CV%	Mean IU/ml	CV%
1	25.2	3.3	25.5	1.4
2	29.7	8.6	29.7	3.4
3	52.3	10.5	52.3	0.8
4	47.9	4.6	47.9	1.7
5	72.7	1.1	72.8	2.8
6	101.6	10.6	101.6	5.7
7	139.7	5.7	139.7	2.7
8	107.5	12.5	107.5	1.6
9	315.0	8.7	315.0	4.2

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DIESSE Diagnostica Senese S.p.A.