

## Premier Affinity A1c 3000

**REF** 09-03-0007

Pour d'autres langues  
Für andere Sprachen  
Para otras lenguas  
Per le altre lingue  
Dla innych języków

Para outras línguas  
Για τις άλλεςλώσσες  
För andra språk  
For andre språk



[www.trinitybiotech.com](http://www.trinitybiotech.com)

### INTENDED USE

This reagent-column kit is a part number for convenience in ordering reagents and column(s) in the approximate ratio used by the Premier Hb9210 HbA1c Analyzer under average testing usage. This is not a test kit and has no lot number. It is not a matched set and may be used in combination with any other reagents or column lots.

The Premier Hb9210 HbA1c Analyzer is 510k cleared and CE marked. The reagents and column(s) are a required accessory for the analyzer. Individual reagents and column(s) are supported with lot numbers, IFUs and CofAs.

For use with the Premier Hb9210 HbA1c System only. No other uses are intended or authorized. No substitutions are authorized or permitted.

### SUMMARY

This reagent-column kit is for use with the Premier Hb9210 HbA1c Analyzer. No preparation is necessary. The reagents and column(s) are ready to use with the analyzer.

### STORAGE AND STABILITY

#### Short-term Transport-Shipping

2 °C 28 °C

The kit may be transported and stored between 2 and 28°C for up to 30 days. For long-term storage the column(s) should be removed and stored at 2 to 8°C for best results. Do not freeze.

#### Long-term Storage of the Column(s)

2 °C 8 °C

For long-term storage of the column(s), remove the column(s) from the kit and store at 2 to 8°C for best results. Do not freeze.

#### Long-term Storage of the Reagents

2 °C 28 °C

For long-term storage of the reagents, remove the column(s) for refrigeration and store the reagents at cool ambient temperatures at 2 to 28°C for best results.

The reagents and column(s) are stable until the expiration date indicated on the item label when kept tightly closed and protected from extreme environmental conditions.



See individual container labels for the expiration date.

Do not use after the expiration date.

### PRECAUTIONS

Consult the individual product safety data sheet (SDS) for safety information.

### CONTENTS OF REAGENT PACK

01-03-0080	Premier Hb9210 Buffer A Reagent, 3.8 L	QTY: 3
01-03-0081	Premier Hb9210 Buffer B Reagent, 3.8 L	QTY: 3
01-03-0097	Premier Hb9210 Diluent reagent, 3.8 L	QTY: 9
01-03-0098	Premier Hb9210 Wash Reagent	QTY: 3
03-11-0150	Frit, 2 Micron	QTY: 3
09-41-1905	Frit, 75 Micron	QTY: 3
09-06-0046	Premier Hb9210 HbA1c Analytical Column	QTY: 3

### ORDERING INFORMATION

REF Number	Item	Quantity
09-03-0007	Premier Affinity A1c 3000	1



Trinity Biotech  
Kansas City, MO 64132 USA  
Tel: 1 800-325-3424  
Fax: 1 816-361-1974



Trinity Biotech plc  
Bray Co. Wicklow, Ireland  
Tel. 353 1 2769800  
Fax 353 1 2769888  
[www.trinitybiotech.com](http://www.trinitybiotech.com)



*PH 21/1/2018*  
*AS 2018/1/13*