

RANSEL CONTROL

CAT. NO. SC 692 **LOT NO.** 524RS
SIZE: 10 x 1 ml **EXPIRY:** 2025-03-28
GTIN: 05055273206135

INTENDED USE

This product is intended for *in vitro* diagnostic use in the control of the Randox Ransel assay.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

This control is manufactured from bovine whole blood. It contains no additives from human sources.

Material safety data sheets available on request.

PREPARATION FOR USE

Reconstitute each vial with exactly 1 ml of distilled water and allow to stand for 30 minutes prior to use. Ransel control should be diluted 1 + 40 prior to analysis (Dilution factor = 41). For example, 50 µl of control should be diluted with 2 ml of diluting agent.

STORAGE AND STABILITY

UNOPENED: Store refrigerated at +2°C to +8°C. Stable to expiry date printed on individual vials.

OPENED: Once reconstituted, the Ransel control is stable for 3 days when stored at +2°C to +8°C.

MATERIALS PROVIDED

Ransel Control 10 x 1 ml

MATERIALS REQUIRED BUT NOT PROVIDED

Randox Ransel Kit, Cat. No. RS 504 or RS 505.

VALUE ASSIGNMENT

Glutathione peroxidase level

U/I = 467 (397 – 537)

EC	REP
----	-----

Randox Teoranta, Meenmore,
Dungloe, Donegal,
F94 TV06, Ireland

19 Oct '23 ld

THIS PAGE IS INTENTIONALLY BLANK